

INTOXILYZER 8000

Location: YCSO - VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

06/13/2017

19:41:00

Standard Lot#: 27016100A1

Last Changed By: J. DUBIEN #09611

QAS: S. CHUPP #9808

YAVAPAI CSO

Operator: S. CHUPP #9808

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	19:41:32
0.100 Cal Check	0.099	19:41:53
Air Blank	0.000	19:42:23

Preventative Maintenance
Performed -- Timer Reset

DR#17-020925

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST SEAN CHUPP AGENCY Y.C.S.O.
DATE 06/13/17 TIME 1943
INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE JAIL

(X) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (X) 2. Date check.

OPERATIONAL TESTS


- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (X) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (b) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 99 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

PS 1+1-800

SIGNATURE

 #9808

INTOXILYZER 8000

Location: YCSO - VERDE JAIL

Serial Number: 80-001222

Core Version: 8105.48

06/13/2017

19:43:22

Standard Lot#: 27016100A1

Last Changed By: J. DUBIEN

#09611

QAS: S. CHUPP

#9808

YAVAPAI CSO

Operator: S. CHUPP

#9808

YAVAPAI CSO

Subject: SQAP,TEST,I

DOB: 01/01/1990

Sex: M

Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	19:44:30
Diagnostic Test	Pass	19:45:05
Air Blank	0.000	19:45:32
0.100 Cal Check	0.099	19:45:53
Air Blank	0.000	19:46:23
Subject Test	0.000	19:46:42
Air Blank	ABT*	19:46:51
Air Blank	RFI**	19:47:08

*Sequence Aborted

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO - VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
06/13/2017 19:49:03

Standard Lot#: 27016100A1
Last Changed By: J. DUBIEN #09611

QAS: S. CHUPP #9808
YAVAPAI CSO

Operator: S. CHUPP #9808
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 01/01/1990
Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	19:49:53
Diagnostic Test	Pass	19:50:28
Air Blank	0.000	19:50:55
0.100 Cal Check	0.099	19:51:16
Air Blank	0.000	19:51:46
Subject Test	XXX*	19:52:07
Air Blank	0.000	19:52:40

*Invalid Sample
Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO - VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
06/13/2017 19:47:44

Standard Lot#: 27016100A1
Last Changed By: J. DUBIEN #09611

QAS: S. CHUPP #9808
YAVAPAI CSO

Operator: S. CHUPP #9808
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 01/01/1990
Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	19:48:28
Air Blank	0.000	19:48:56

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YCSO - VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
06/13/2017 19:53:11

Standard Lot#: 27016100A1
Last Changed By: J. DUBIEN #09611

QAS: S. CHUPP #9808
YAVAPAI CSO

Operator: S. CHUPP #9808
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 01/01/1990
Sex: M Weight: 180

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	19:54:11
Diagnostic Test	Pass	19:54:46
Air Blank	0.000	19:55:13
0.100 Cal Check	0.099	19:55:34
Air Blank	0.000	19:56:04
Subject Test	0.058*	19:59:11
Air Blank	0.000	19:59:43
Air Blank	0.000	20:00:10
Subject Test	ABT**	20:00:19
Air Blank	0.000	20:00:47

*Deficient Sample
**Sequence Aborted

Not a Successfully Completed Test Sequence

DRA 17-020925

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST SOAN CHUPP AGENCY Y.C.S.O.


DATE 06/13/17 TIME 2001

INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE JAIL

(X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR

~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~

- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 99 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  # 9808

INTOXILYZER 8000
Location: YCSO - VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
06/13/2017 20:01:16

Standard Lot#: 27016100A1
Last Changed By: J. DUBIEN #09611

QAS: S. CHUPP #9808
YAVAPAI CSO

Operator: S. CHUPP #9808
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	20:01:47
0.100 Cal Check	0.099	20:02:08
Air Blank	0.000	20:02:37

Preventative Maintenance
Performed -- Timer Reset