

Dr 17-059092

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)


ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST SEAN CHUPP AGENCY Y.C.S.O.

DATE 11/1/17 TIME 1625

INTOXILYZER SERIAL # 80-001222 LOCATION CAMP VERDE SAIL

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC. into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (x) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (>) 4. Air blank completed.
- (>) 5. Calibration check completed. Test results 0. 098 AC.
- (>) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  300PS1

INTOXILYZER 8000
Location: YCSO - VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
11/01/2017 16:25:30

Standard Lot#: 27016100A1
Last Changed By: J. DUBIEN #09611

QAS: S. CHUPP #9808
YAVAPAI CSO

Operator: S. CHUPP #9808
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	16:26:03
0.100 Cal Check	0.098	16:26:24
Air Blank	0.000	16:26:53

Preventative Maintenance
Performed -- Timer Reset