

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST MCQUEARY AGENCY YCSO
DATE 6/30/18 TIME 1549
INTOXILYZER SERIAL # 80-001222 LOCATION CV JAIL

- (K) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (K) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 099 AC.
- (X) 6. Air blank completed.
- (K) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]
DPS Form Exh G-2 (Rev 05-01)

18-023996

INTOXILYZER 8000
Location: YCSO - VERDE JAIL
Serial Number: 80-001222
Core Version: 8105.48
06/30/2018 15:49:11

Standard Lot#: 06717100A5
Last Changed By: J. MCQUEARY #E54

QAS: J. MCQUEARY #E54
YAVAPAI CSO

Operator: J. MCQUEARY #E54
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	15:50:09
0.100 Cal Check	0.099	15:50:30
Air Blank	0.000	15:51:00

Preventative Maintenance
Performed -- Timer Reset