

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co. S.O.  
DATE 5-28-08 TIME 1445  
INTOXILYZER SERIAL # 80-001455 LOCATION YCSO-BAGDAD

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0. 000 AC.
- 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 100 AC. Result 0. 102 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS RESET DATE/CLOCK. DATE/TIME OFF.

SIGNATURE Daniel Q Raiss #5-31/OAS #33026

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number : 80-001455

05/28/2008

15:07:45

Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535

QAS: D. RAISS #3693

YAVAPAI CSO

Operator: D. RAISS #3693

YAVAPAI CSO

Subject: SQAP, TEST,

DOB: 01/01/1901

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:08:40
Diagnostic Test	Pass	15:09:08
Air Blank	0.000	15:09:35
0.100 Cal Check	0.102	15:09:54
Air Blank	0.000	15:10:23
Subject Test	0.055*	15:13:31
Air Blank	0.000	15:14:03
Air Blank	0.000	15:14:30
Subject Test	XXX**	15:14:58
Air Blank	0.000	15:15:31

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number : 80-001455

05/28/2008

15:06:17

Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535

QAS: D. RAISS #3693

YAVAPAI CSO

Operator: D. RAISS #3693

YAVAPAI CSO

Subject: SQAP,TEST,

DOB: 01/01/1901

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	15:06:53
Air Blank	0.000	15:07:21

\*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number : 80-001455

05/28/2008

15:04:39

Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535

QAS: D. RAISS #3693

YAVAPAI CSO

Operator: D. RAISS #3693

YAVAPAI CSO

Subject: SQAP,TEST,

DOB: 01/01/1901

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	15:05:29
Air Blank	0.000	15:05:57

\*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number : 80-001455

05/28/2008

15:00:22

Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535

QAS: D. RAISS #3693

YAVAPAI CSO

Operator: D. RAISS #3693

YAVAPAI CSO

Subject: SQAP,TEST,

DOB: 01/01/1901

Sex: M

Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:01:13
Diagnostic Test	Pass	15:01:41
Air Blank	0.000	15:02:08
0.100 Cal Check	0.101	15:02:27
Air Blank	0.000	15:02:56
Subject Test	0.000	15:03:28
Air Blank	0.000	15:03:56
Five Minute Wait	ABT*	
Air Blank	0.000	15:04:31

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co. S.O.  
DATE 5-28-08 TIME 1445  
INTOXILYZER SERIAL # 80-001455 LOCATION YCSO-BAGDAD

- (02) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (02) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (02) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (02) 4. Air blank completed.
- (02) 5. Calibration check completed. Test results 0. 102 AC.
- (02) 6. Air blank completed.
- (02) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Daniel Q. Raiss #5-31

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

05/28/2008

15:16:42

-----  
Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535  
-----

QAS: D. RAISS #3693

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	15:17:09
Cal Check	0.102	15:17:28
Air Blank	0.000	15:17:57

Amil Q. Rains #5-31/OAS #33026  
Operator's Signature