

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO
DATE 9.23.09 TIME 0904
INTOXILYZER SERIAL # 80-001455 LOCATION Bagdad Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Deann Ashby #562/QAS 33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

09/23/2009

09:03:45

Standard Lot#: 730901I

Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	09:04:12
Cal Check	0.099	09:04:31
Air Blank	0.000	09:05:00

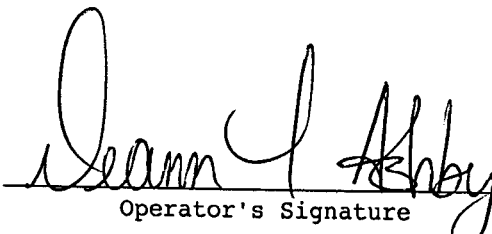

Operator's Signature #S62/QAS #33400

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. Ashby AGENCY YCSO

DATE 9.23.09 TIME 0906

INTOXILYZER SERIAL # 80-001455 LOCATION Bagdad Substation

(✓) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Deann L Ashby #562/QAS#33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number : 80-001455

09/23/2009

09:05:52

Standard Lot#: 730901I

Last Changed By: D. ASHBY

#S62

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:06:37
Diagnostic Test	Pass	09:07:05
Air Blank	0.000	09:07:32
0.100 Cal Check	0.099	09:07:51
Air Blank	0.000	09:08:20
Subject Test	0.000	09:08:52
Air Blank	0.000	09:09:19
Five Minute Wait	ABT*	
Air Blank	0.000	09:09:53

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number : 80-001455

09/23/2009

09:09:58

Standard Lot#: 730901I

Last Changed By: D. ASHBY

#S62

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	09:10:30
Air Blank	0.000	09:10:58

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YAVAPAI SO, BAGDAD
Serial Number : 80-001455
09/23/2009 09:11:03

Standard Lot#: 730901I
Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62
YAVAPAI CSO

Operator: D. ASHBY #S62
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 01/01/1999
Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	09:11:29
Air Blank	0.000	09:11:57

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YAVAPAI SO, BAGDAD
Serial Number : 80-001455
09/23/2009

09:12:02

Standard Lot#: 730901I
Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62
YAVAPAI CSO

Operator: D. ASHBY #S62
YAVAPAI CSO

Subject: SQAP, TES
DOB: 01/01/1999
Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:12:45
Diagnostic Test	Pass	09:13:13
Air Blank	0.000	09:13:40
0.100 Cal Check	0.099	09:13:59
Air Blank	0.000	09:14:28
Subject Test	0.122*	09:17:36
Air Blank	0.000	09:18:11
Air Blank	0.000	09:18:38
Subject Test	XXX**	09:19:20
Air Blank	0.000	09:19:54

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO
DATE 9.23.09 TIME 0920
INTOXILYZER SERIAL # 80-001455 LOCATION Bagdad Substation

(✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration check completed. Test results 0.098 AC.
(✓) 6. Air blank completed.
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Dann Ashby #562 / QAS # 33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

09/23/2009

09:20:16

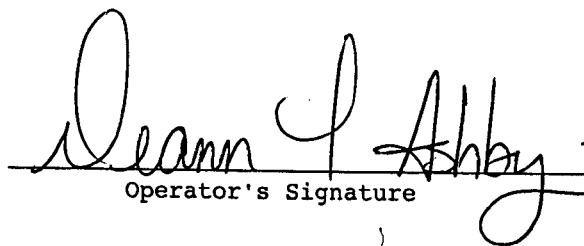
Standard Lot#: 730901I

Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	09:20:43
Cal Check	0.098	09:21:02
Air Blank	0.000	09:21:31


Operator's Signature #S62 / QAS #33400