

RECEIVED
DEC 03 2009
YAVAPAI COUNTY ATTORNEY

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #5162 AGENCY YCSO
DATE 11-24-09 TIME 1620
INTOXILYZER SERIAL # 08-001455 LOCATION Bagdad Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration check completed. Test results 0. 100 AC.
(✓) 6. Air blank completed.
(✓) 7. Remove printed report. Attach the record to the completed checklist.

SIGNATURE D. Ashby

DPS Form Exh G-2 (Rev 05-01)

- Tank expired 11/6/09
- I changed expiration to show 11/25/09 so that I could complete the Standard Calibration
- New tank to be changed out on 11/25/09

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

11/24/2009

16:20:45


Standard Lot#: 730901I

Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	16:21:12
Cal Check	0.100	16:21:31
Air Blank	0.000	16:22:00

 #S62 / QAS#33400
Operator's Signature