

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. ASHBY #562 AGENCY YCSO

DATE 1-26-10 TIME 1615

INTOXILYZER SERIAL # 80-001455 LOCATION BAGDAD SUBSTATION

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 101 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Deann Ashby #562 / QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

01/26/2010

16:15:54

Standard Lot#: 585220

Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62

YAVAPAI CSO

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | 0.000 | 16:16:21 |
| Cal Check | 0.101 | 16:16:40 |
| Air Blank | 0.000 | 16:17:09 |

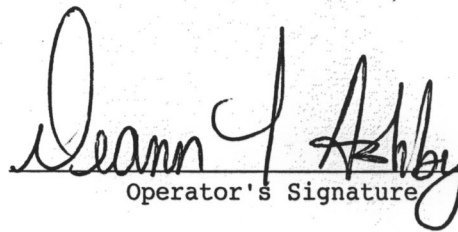
 #S62/QAS #33400
Operator's Signature

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. ASHBY #562 AGENCY YCSO

DATE 1.26.10 TIME 1617

INTOXILYZER SERIAL # 80-001455 LOCATION BAGDAD SUBSTATION

(✓) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

(✓) 1. Clock time check.

(✓) 2. Date check.

OPERATIONAL TESTS

(✓) 1. Alcohol-free subject test result 0. 000 AC.

(✓) 2. Error recognition logic system functioning.

Not a Successfully Completed Test Sequence printed

(✓) 3. Proper sample recognition system.

Not a Successfully Completed Test Sequence printed

Deficient sample printed.

(✓) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE D. Ashby #562 / QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number : 80-001455

01/26/2010

16:17:14

Standard Lot#: 585220

Last Changed By: D. ASHBY

#S62

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP, TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|------------------|--------|----------|
| Air Blank | 0.000 | 16:18:05 |
| Diagnostic Test | Pass | 16:18:32 |
| Air Blank | 0.000 | 16:18:59 |
| 0.100 Cal Check | 0.100 | 16:19:18 |
| Air Blank | 0.000 | 16:19:47 |
| Subject Test | 0.000 | 16:20:14 |
| Air Blank | 0.000 | 16:20:41 |
| Five Minute Wait | ABT* | |
| Air Blank | 0.000 | 16:21:16 |

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number : 80-001455

01/26/2010

16:21:20

Standard Lot#: 585220

Last Changed By: D. ASHBY

#S62

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP, TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | IPS* | 16:21:50 |
| Air Blank | 0.000 | 16:22:18 |

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number : 80-001455

01/26/2010

16:23:00

Standard Lot#: 585220

Last Changed By: D. ASHBY

#S62

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | RFI* | 16:23:31 |
| Air Blank | 0.000 | 16:23:59 |

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number : 80-001455

01/26/2010

16:24:17

Standard Lot#: 585220

Last Changed By: D. ASHBY

#S62

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 16:25:04 |
| Diagnostic Test | Pass | 16:25:32 |
| Air Blank | 0.000 | 16:25:59 |
| 0.100 Cal Check | 0.099 | 16:26:18 |
| Air Blank | 0.000 | 16:26:47 |
| Subject Test | 0.099* | 16:29:55 |
| Air Blank | 0.000 | 16:30:29 |
| Air Blank | 0.000 | 16:30:57 |
| Subject Test | XXX** | 16:31:18 |
| Air Blank | 0.000 | 16:31:55 |

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. ASHBY #562 AGENCY YCSO

DATE 1-26-10 TIME 1632

INTOXILYZER SERIAL # 80-001455 LOCATION BAGDAD SUBSTATION

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 097 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Dawn Ashby #562/QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

01/26/2010

16:32:50


Standard Lot#: 585220

Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62

YAVAPAI CSO

| Test | g/210L | Time |
|-----------|--------|----------|
| Air Blank | 0.000 | 16:33:18 |
| Cal Check | 0.097 | 16:33:37 |
| Air Blank | 0.000 | 16:34:06 |

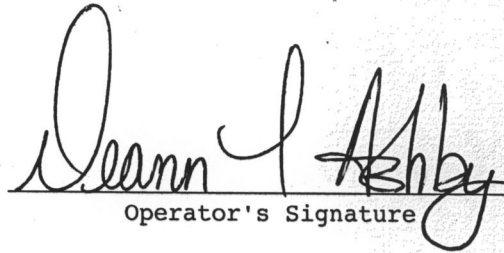 #S62/QAS #33400
Operator's Signature