

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co. S.O.  
DATE 2-24-10 TIME 1300  
INTOXILYZER SERIAL # 80-001455 LOCATION YCSO-BAGDAD

- (OR) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
Pour a standard alcohol concentration solution X OR \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (OR) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (OR) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (OR) 4. Air blank completed.
- (OR) 5. Calibration check completed. Test results 0. 101 AC.
- (OR) 6. Air blank completed.
- (OR) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Amiel Q Rains #N-11 / QAS #33026

DPS Form Exh G-2 (Rev 05-01)

SOFTWARE UPGRADE.  
CAL. CK ALL OK.

INTOXILYZER 8000  
Location: YAVAPAI SO, BAGDAD  
Serial Number: 80-001455  
Core Version: 8105.48  
02/24/2010 13:19:03

Standard Lot#: 585220  
Last Changed By: D. ASHBY #562

DAS: D. RAISS #3693  
YAVAPAI CSO

Operator: D. RAISS #3693  
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	13:19:32
0.100 Cal Check	0.101	13:19:53
Air Blank	0.000	13:20:23

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST DAN RAIS AGENCY YAVAPAI Co. S.O.  
DATE 2-24-10 TIME 1300  
INTOXILYZER SERIAL # 80-001455 LOCATION YCSO - BAGDAD

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.  
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.  
 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 101 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

SOFTWARE UPGRADE. SAME DRY GAS. ALL OK.

SIGNATURE

Daniel Q Rains #N-11 / QAS #33026

INTOXILYZER 8000  
Location: YAUAPAI SQ, BACDPA0  
Serial Number: 80-001455  
Core Version: 8105.48  
02/24/2010 13:03:34

Standard Lot#: 585220  
Last Changed By: D. ASHBY #562  
OAS: D. RAJISS #3693  
YAUAPAI CS0

Operator: D. RAJISS #3693  
YAUAPAI CS0  
Subject: SQAP TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:05:01
Diagnostic Test	Pass	13:05:38
Air Blank	0.000	13:06:06
0.100 Cal Check	0.100	13:06:27
Air Blank	0.000	13:06:56
Subject Test	0.000	13:07:34
Air Blank	0.000	13:08:02
AST	AST*	
Air Blank	0.000	13:08:42

\*Sequence Abortec

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: YAUAPAI SQ, BACDPA0  
Serial Number: 80-001455  
Core Version: 8105.48  
02/24/2010 13:08:50

Standard Lot#: 585220  
Last Changed By: D. ASHBY #562  
OAS: D. RAJISS #3693  
YAUAPAI CS0

Operator: D. RAJISS #3693  
YAUAPAI CS0  
Subject: SQAP TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:09:13
Air Blank	0.000	13:09:41

\*Improper Sample

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: YAUAPAI SQ, BACDPA0  
Serial Number: 80-001455  
Core Version: 8105.48  
02/24/2010 13:09:55

Standard Lot#: 585220  
Last Changed By: D. ASHBY #562  
OAS: D. RAJISS #3693  
YAUAPAI CS0

Operator: D. RAJISS #3693  
YAUAPAI CS0  
Subject: SQAP TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:10:24
Air Blank	0.000	13:10:52

\*RFI detect

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: YAUAPAI SQ, BACDPA0  
Serial Number: 80-001455  
Core Version: 8105.48  
02/24/2010 13:11:10

Standard Lot#: 585220  
Last Changed By: D. ASHBY #562  
OAS: D. RAJISS #3693  
YAUAPAI CS0

Operator: D. RAJISS #3693  
YAUAPAI CS0  
Subject: SQAP TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:11:49
Diagnostic Test	Pass	13:12:26
Air Blank	0.000	13:12:54
0.100 Cal Check	0.100	13:13:15
Air Blank	0.000	13:13:44
Subject Test	0.127*	13:16:52
Air Blank	0.000	13:17:23
Air Blank	0.000	13:17:51
Subject Test	XXXX*	13:18:15
Air Blank	0.000	13:18:48

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

Not a Successfully  
Completed  
Test Sequence

**EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)**

**ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000**

**STANDARD CALIBRATION CHECK PROCEDURE**

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co. S.O.  
 DATE 2-24-10 TIME 1109  
 INTOXILYZER SERIAL # 80-001455 LOCATION YCSO-BAGDAD

- (OR) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
 Pour a standard alcohol concentration solution OR AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (OR) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (OR) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (OR) 4. Air blank completed.
- (OR) 5. Calibration check completed. Test results 0. 100 AC.
- (OR) 6. Air blank completed.
- (OR) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Daniel Q. Raiss #N-11 / QAS #33026

DPS Form Exh G-2 (Rev 05-01)

SOFTWARE UPGRADE  
CAL CK. ALL OK

INTOXILYZER 8000  
Instrument Initialization  
10:49:24 02/24/2010

INTOXILYZER 8000  
 Location: YAVAPAI SO, BAGDAD  
 Serial Number: 80-001455  
 Core Version: 8105.48  
 02/24/2010 11:08:55

-----  
 Standard Lot#: 585220  
 Last Changed By: D. ASHBY #562

-----  
 QAS: D. RAISS #3693  
 YAVAPAI CSO

-----  
 Operator: D. RAISS #3693  
 YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	11:09:25
0.100 Cal Check	0.100	11:09:46
Air Blank	0.000	11:10:15

Preventative Maintenance  
Performed -- Timer Reset