

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO
DATE 7-23-10 TIME 0658
INTOXILYZER SERIAL # 80-001455 LOCATION Bagdad Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

Dawn Y Ashby #562 / QAS#33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

07/23/2010

06:58:03

Standard Lot#: 585220

Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62

YAVAPAI CSO

Operator: D. ASHBY #S62

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	06:58:50
0.100 Cal Check	0.100	06:59:11
Air Blank	0.000	06:59:41

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO
DATE 7-23-10 TIME 0703

INTOXILYZER SERIAL # 80-001455 LOCATION Bagdad Substation

(✓) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
(✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
(✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(✓) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

Aborted 1st test because I put my 1st name as
subject.

SIGNATURE Deann J Ashby #562 / QAS # 33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

07/23/2010

07:03:23

Standard Lot#: 585220

Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62

YAVAPAI CSO

Operator: D. ASHBY #S62

YAVAPAI CSO

Subject: SQAP, DEANN

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:04:46
Diagnostic Test	Pass	07:05:20
Air Blank	ABT*	07:05:34
Air Blank	0.000	07:06:03

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

07/23/2010

07:06:10

Standard Lot#: 585220

Last Changed By: D. ASHBY

#S62

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:07:15
Diagnostic Test	Pass	07:07:49
Air Blank	0.000	07:08:17
0.100 Cal Check	0.100	07:08:38
Air Blank	0.000	07:09:08
Subject Test	0.000	07:09:55
Air Blank	0.000	07:10:22
Wait	ABT*	
Air Blank	0.000	07:10:57

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

07/23/2010

07:11:09

Standard Lot#: 585220

Last Changed By: D. ASHBY

#S62

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	07:11:42
Air Blank	0.000	07:12:11

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YAVAPAI SO, BAGDAD
Serial Number: 80-001455
Core Version: 8105.48
07/23/2010 07:12:52

Standard Lot#: 585220
Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62
YAVAPAI CSO

Operator: D. ASHBY #S62
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 01/01/1999
Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	07:13:56
Air Blank	0.000	07:14:24

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YAVAPAI SO, BAGDAD
Serial Number: 80-001455
Core Version: 8105.48
07/23/2010 07:18:49

Standard Lot#: 585220
Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62
YAVAPAI CSO

Operator: D. ASHBY #S62
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 01/01/1999
Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:19:49
Diagnostic Test	Pass	07:20:23
Air Blank	0.000	07:20:51
0.100 Cal Check	0.099	07:21:11
Air Blank	0.000	07:21:41
Subject Test	0.091*	07:24:49
Air Blank	0.000	07:25:28
Air Blank	0.000	07:25:56
Subject Test	XXX**	07:26:41
Air Blank	0.000	07:27:15

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO

DATE 7.23.10 TIME 0728

INTOXILYZER SERIAL # 80.001455 LOCATION Bagdad Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.098 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

Deann Y Ashby #562/QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

07/23/2010

07:28:48

Standard Lot#: 585220

Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62

YAVAPAI CSO

Operator: D. ASHBY #S62

YAVAPAI CSO

Test g/210L Time

Air Blank 0.000 07:29:20
0.100 Cal Check 0.098 07:29:41
Air Blank 0.000 07:30:11

Preventative Maintenance
Performed -- Timer Reset