

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO  
DATE 8-23-10 TIME 1023  
INTOXILYZER SERIAL # 80-001455 LOCATION Bagdad Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Deann J Ashby #562 / QAS#33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

08/23/2010

10:23:25

-----  
Standard Lot#: 585220

Last Changed By: D. ASHBY #S62  
-----

QAS: D. ASHBY #S62

YAVAPAI CSO  
-----

Operator: D. ASHBY #S62

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	10:24:39
0.100 Cal Check	0.099	10:24:59
Air Blank	0.000	10:25:29

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY Y050  
DATE 8-23-10 TIME 1025  
INTOXILYZER SERIAL # 80-001455 LOCATION Bogdad Substation

(✓) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Deann I Ashby #562 / QAS # 33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

08/23/2010

10:25:45

---

Standard Lot#: 585220

Last Changed By: D. ASHBY #S62

---

QAS: D. ASHBY #S62

YAVAPAI CSO

---

Operator: D. ASHBY #S62

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	10:26:37
Diagnostic Test	Pass	10:27:12
Air Blank	0.000	10:27:39
0.100 Cal Check	0.100	10:28:00
Air Blank	0.000	10:28:30
Subject Test	0.000	10:29:01
Air Blank	0.000	10:29:29
Wait	ABT*	
Air Blank	0.000	10:30:26

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YAVAPAI SO, BAGDAD  
Serial Number: 80-001455  
Core Version: 8105.48  
08/23/2010 10:31:57

---

Standard Lot#: 585220  
Last Changed By: D. ASHBY #S62

---

QAS: D. ASHBY #S62  
YAVAPAI CSO

---

Operator: D. ASHBY #S62  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	10:32:55
Air Blank	0.000	10:33:23

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

08/23/2010

10:34:11

---

Standard Lot#: 585220

Last Changed By: D. ASHBY

#S62

---

QAS: D. ASHBY

#S62

YAVAPAI CSO

---

Operator: D. ASHBY

#S62

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	10:34:51
Air Blank	0.000	10:35:20

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

08/23/2010

10:37:55

---

Standard Lot#: 585220

Last Changed By: D. ASHBY

#S62

---

QAS: D. ASHBY

#S62

YAVAPAI CSO

---

Operator: D. ASHBY

#S62

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	10:38:49
Diagnostic Test	Pass	10:39:24
Air Blank	0.000	10:39:52
0.100 Cal Check	0.100	10:40:12
Air Blank	0.000	10:40:42
Subject Test	0.103*	10:43:50
Air Blank	0.000	10:44:22
Air Blank	0.000	10:44:49
Subject Test	XXX**	10:45:19
Air Blank	0.000	10:45:54

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO

DATE 8-23-10 TIME 1046

INTOXILYZER SERIAL # 80-001455 LOCATION Bagdad Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Deann J Ashby #562/QAS#33400

INTOXILYZER 8000  
Location: YAVAPAI SO, BAGDAD  
Serial Number: 80-001455  
Core Version: 8105.48  
08/23/2010 10:46:32

-----  
Standard Lot#: 585220  
Last Changed By: D. ASHBY #S62  
-----

QAS: D. ASHBY #S62  
YAVAPAI CSO  
-----

Operator: D. ASHBY #S62  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	10:47:02
0.100 Cal Check	0.099	10:47:23
Air Blank	0.000	10:47:53

Preventative Maintenance  
Performed -- Timer Reset