

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #5162 AGENCY YCSO  
DATE 9-11-12 TIME 0753  
INTOXILYZER SERIAL # 80-001455 LOCATION Bagdad Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Deann J Ashby #5162/QAS#33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

09/11/2012

07:53:33

-----  
Standard Lot#: 03012100A1

Last Changed By: D. ASHBY #S62  
-----

QAS: D. ASHBY #S62

YAVAPAI CSO  
-----

Operator: D. ASHBY #S62

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	07:54:03
0.100 Cal Check	0.100	07:54:24
Air Blank	0.000	07:54:54

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. Ashby #5162 AGENCY YCSO  
DATE 9.11.12 TIME 0755  
INTOXILYZER SERIAL # 80.001455 LOCATION Bagdad Substation

(✓) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.  
(✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.  
(✓) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
(✓) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(✓) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

---

---

---

---

SIGNATURE Deann J Ashby #5162 / QAS #33400

INTOXILYZER 8000  
Location: YAVAPAI SO, BAGDAD  
Serial Number: 80-001455  
Core Version: 8105.48  
09/11/2012

07:55:02

---

Standard Lot#: 03012100A1  
Last Changed By: D. ASHBY #S62

---

QAS: D. ASHBY #S62  
YAVAPAI CSO

---

Operator: D. ASHBY #S62  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	07:55:49
Diagnostic Test	Pass	07:56:24
Air Blank	0.000	07:56:52
0.100 Cal Check	0.101	07:57:12
Air Blank	0.000	07:57:42
Subject Test	0.000	07:58:15
Air Blank	0.000	07:58:42
Wait	ABT*	
Air Blank	0.000	07:59:26

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

09/11/2012

07:59:31

---

Standard Lot#: 03012100A1

Last Changed By: D. ASHBY #S62

---

QAS: D. ASHBY #S62

YAVAPAI CSO

---

Operator: D. ASHBY #S62

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	08:00:30
Air Blank	0.000	08:00:59

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

09/11/2012

08:02:04

---

Standard Lot#: 03012100A1

Last Changed By: D. ASHBY

#S62

---

QAS: D. ASHBY

#S62

YAVAPAI CSO

---

Operator: D. ASHBY

#S62

YCSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	08:03:05
Air Blank	0.000	08:03:33

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YAVAPAI SO, BAGDAD  
Serial Number: 80-001455  
Core Version: 8105.48  
09/11/2012

08:03:41

---

Standard Lot#: 03012100A1  
Last Changed By: D. ASHBY #S62

---

QAS: D. ASHBY #S62  
YAVAPAI CSO

---

Operator: D. ASHBY #S62  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	08:04:51
Diagnostic Test	Pass	08:05:26
Air Blank	0.000	08:05:53
0.100 Cal Check	0.100	08:06:14
Air Blank	0.000	08:06:44
Subject Test	0.120*	08:09:51
Air Blank	0.000	08:10:24
Air Blank	0.000	08:10:52
Subject Test	XXX**	08:11:28
Air Blank	0.000	08:12:02

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO

DATE 9.11.12 TIME 0812

INTOXILYZER SERIAL # 80-001455 LOCATION Bagdad Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Dann Ashby #562 / QAS#33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

09/11/2012

08:12:54

-----  
Standard Lot#: 03012100A1

Last Changed By: D. ASHBY #S62  
-----

QAS: D. ASHBY #S62

YAVAPAI CSO  
-----

Operator: D. ASHBY #S62

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	08:13:24
0.100 Cal Check	0.099	08:13:44
Air Blank	0.000	08:14:15

Preventative Maintenance  
Performed -- Timer Reset