

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #2271 AGENCY YCSO
DATE 3-19-13 TIME 0906
INTOXILYZER SERIAL # 80-001455 LOCATION Bagdad Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution of 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Deann P Ashby #2271 / QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

03/19/2013

09:06:38

Standard Lot#: 13712100A2

Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62

YAVAPAI CSO

Operator: D. ASHBY #S62

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	09:07:15
0.100 Cal Check	0.100	09:07:36
Air Blank	0.000	09:08:06

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. Ashby #2271 AGENCY YCSD
DATE 3-19-13 TIME 0910
INTOXILYZER SERIAL # 80.001455 LOCATION Bagdad Substation

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
() 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
() 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

RFI Detect when I set my phone next
to the machine during the "Deficient Sample"
test.

SIGNATURE Deann J Ashby #2271 / QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

03/19/2013

09:10:41

Standard Lot#: 13712100A2

Last Changed By: D. ASHBY

#S62

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:12:24
Diagnostic Test	Pass	09:12:59
Air Blank	0.000	09:13:26
0.100 Cal Check	0.100	09:13:47
Air Blank	0.000	09:14:17
Subject Test	0.000	09:14:50
Air Blank	0.000	09:15:17
Wait	ABT*	
Air Blank	0.000	09:16:13

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YAVAPAI SO, BAGDAD
Serial Number: 80-001455
Core Version: 8105.48
03/19/2013

09:28:21

Standard Lot#: 13712100A2
Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62
YAVAPAI CSO

Operator: D. ASHBY #S62
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 01/01/1999
Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	09:28:55
Air Blank	0.000	09:29:23

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YAVAPAI SO, BAGDAD
Serial Number: 80-001455
Core Version: 8105.48
03/19/2013

09:29:36

Standard Lot#: 13712100A2
Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62
YAVAPAI CSO

Operator: D. ASHBY #S62
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 01/01/1999
Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	09:30:18
Air Blank	0.000	09:30:46

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YAVAPAI SO, BAGDAD
Serial Number: 80-001455
Core Version: 8105.48
03/19/2013

09:30:58

Standard Lot#: 13712100A2
Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62
YAVAPAI CSO

Operator: D. ASHBY #S62
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 01/01/1999
Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:31:43
Diagnostic Test	Pass	09:32:18
Air Blank	0.000	09:32:46
0.100 Cal Check	0.099	09:33:06
Air Blank	0.000	09:33:36
Subject Test	0.223*	09:36:44
Air Blank	0.000	09:37:16
Air Blank	0.000	09:37:44
Subject Test	RFI**	09:38:07
Air Blank	0.000	09:38:35

*Deficient Sample
**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

03/19/2013

09:39:26

Standard Lot#: 13712100A2

Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62

YAVAPAI CSO

Operator: D. ASHBY #S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:40:15
Diagnostic Test	Pass	09:40:50
Air Blank	0.000	09:41:18
0.100 Cal Check	0.099	09:41:38
Air Blank	0.000	09:42:08
Subject Test	0.137*	09:45:16
Air Blank	0.000	09:45:49
Air Blank	0.000	09:46:16
Subject Test	XXX**	09:46:40
Air Blank	0.000	09:47:14

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #2271 AGENCY YCSO
DATE 3.19.13 TIME 0947
INTOXILYZER SERIAL # 80.00455 LOCATION Bagdad Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Deann Ashby #2271 / QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

03/19/2013

09:47:28

Standard Lot#: 13712100A2

Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62

YAVAPAI CSO

Operator: D. ASHBY #S62

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	09:47:58
0.100 Cal Check	0.099	09:48:19
Air Blank	0.000	09:48:49

Preventative Maintenance
Performed -- Timer Reset