

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #2271 AGENCY YCSO
DATE 9.28.15 TIME 1941
INTOXILYZER SERIAL # 80-001455 LOCATION Bagdad Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 097 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

D. Ashby #2271 / QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

09/28/2015

19:41:31

Standard Lot#: 26614100A6

Last Changed By: D. ASHBY #2271

QAS: D. ASHBY #2271

YAVAPAI CSO

Operator: D. ASHBY #2271

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	19:42:11
0.100 Cal Check	0.097	19:42:32
Air Blank	0.000	19:43:01

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. Ashby #2271 AGENCY YCSO

DATE 9.28.15 TIME 1944

INTOXILYZER SERIAL # 80.001455 LOCATION Bagdad Substation

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.

2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.

2. Error recognition logic system functioning.

Not a Successfully Completed Test Sequence printed

3. Proper sample recognition system.

Not a Successfully Completed Test Sequence printed

Deficient sample printed.

4. Standard Calibration Check standard 0. 100 AC. Result 0. 094 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

Went past the 30 days as I was on vacation. Instrument is working properly.

SIGNATURE D. Ashby #2271 / QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

09/28/2015

19:44:42

Standard Lot#: 26614100A6

Last Changed By: D. ASHBY

#2271

QAS: D. ASHBY

#2271

YAVAPAI CSO

Operator: D. ASHBY

#2271

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	19:45:29
Diagnostic Test	Pass	19:46:04
Air Blank	0.000	19:46:31
0.100 Cal Check	0.097	19:46:52
Air Blank	0.000	19:47:21
Subject Test	0.000	19:47:37
Air Blank	0.000	19:48:05
Wait	ABT*	
Air Blank	0.000	19:48:42

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

09/28/2015

19:48:52

Standard Lot#: 26614100A6

Last Changed By: D. ASHBY

#2271

QAS: D. ASHBY

#2271

YAVAPAI CSO

Operator: D. ASHBY

#2271

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	19:49:18
Air Blank	0.000	19:49:47

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

09/28/2015

19:50:00

Standard Lot#: 26614100A6

Last Changed By: D. ASHBY

#2271

QAS: D. ASHBY

#2271

YAVAPAI CSO

Operator: D. ASHBY

#2271

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	19:50:42
Air Blank	0.000	19:51:10

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

09/28/2015

19:52:24

Standard Lot#: 26614100A6

Last Changed By: D. ASHBY

#2271

QAS: D. ASHBY

#2271

YAVAPAI CSO

Operator: D. ASHBY

#2271

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	19:53:10
Diagnostic Test	Pass	19:53:45
Air Blank	0.000	19:54:12
0.100 Cal Check	0.097	19:54:33
Air Blank	0.000	19:55:02
Subject Test	0.078*	19:58:10
Air Blank	0.000	19:58:41
Air Blank	0.000	19:59:09
Subject Test	XXX**	19:59:32
Air Blank	0.000	*20:00:05

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby # 2271 AGENCY YCSO
DATE 9.28.15 TIME 2000
INTOXILYZER SERIAL # 80.001455 LOCATION Bagdad Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration check completed. Test results 0. 094 AC.
(✓) 6. Air blank completed.
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

Deann J Ashby # 2271 / QAS # 33400

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

09/28/2015

20:00:19

Standard Lot#: 26614100A6

Last Changed By: D. ASHBY #2271

QAS: D. ASHBY #2271

YAVAPAI CSO

Operator: D. ASHBY #2271

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	20:00:49
0.100 Cal Check	0.094	20:01:10
Air Blank	0.000	20:01:39

Preventative Maintenance
Performed -- Timer Reset