

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST R. Rivera AGENCY YAVAPAI CO SO
DATE 2-1-17 TIME 1731
INTOXILYZER SERIAL # 80-001455 LOCATION Flagstaff

- 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0.096 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INFORMIYZER 8000

Location: YAVAPAI SQ, PACDAP

Serial Number: 80-001415

Core Version: 8105.48

02/01/2017

17:01:39

Standard Lot#: 15615100A1

Last Changed By: R. RIVERA #6356

OAS: R. RIVERA #6356

YAVAPAI CSO

Operator: R. RIVERA #6356

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	17:32:09
0.100 Cal Check	0.096	17:32:10
Air Blank	0.000	17:32:55

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST R. Rivera AGENCY VAUQUAI CO. 50
DATE 2-1-17 TIME 1718
INTOXILYZER SERIAL # 80-001455 LOCATION Bagdad

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

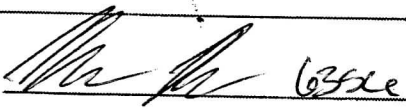
- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0. 000 AC:
- 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 100 AC. Result 0. 096 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE 

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

02/01/2017

17:18:07

Standard Lot#: 15615100A1

Last Changed By: R. RIVERA

#6356

QAS: R. RIVERA

#6356

YAVAPAI CSO

Operator: R. RIVERA

#6356

YAVAPAI CSO

Subject: SQAP, TEST

DOB: 07/18/1980

Sex: M

Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:19:12
Diagnostic Test	Pass	17:19:47
Air Blank	0.000	17:20:15
0.100 Cal Check	0.098	17:20:35
Air Blank	0.000	17:21:05
Subject Test	0.000	17:21:21
Air Blank	ABT*	17:21:30
Air Blank	IPS**	17:21:38

*Sequence Aborted

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
 Location: YAVAPAI SO, BAGDAD
 Serial Number: 80-001455
 Core Version: 8105.48
 02/01/2017 17:21:51

Standard Lot#: 15615100A1
 Last Changed By: R. RIVERA #6356

OAS: R. RIVERA #6356
 YAVAPAI CSO

Operator: R. RIVERA #6356
 YAVAPAI CSO

Subject: SQAP,TEST
 DOB: 07/18/1980
 Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:22:47
Diagnostic Test	Pass	17:23:22
Air Blank	0.000	17:23:49
0.100 Cal Check	0.098	17:24:10
Air Blank	0.000	17:24:39
Subject Test	0.340*	17:27:47
Air Blank	0.000	17:28:20
Air Blank	0.000	17:28:47
Subject Test	XXX**	17:29:11
Air Blank	RFI***	17:29:23

*Deficient Sample

**Invalid Sample

Begin new deprivation period

***RFI Detect

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST R. Rivera AGENCY YAVAPAI CO SO

DATE 2-01-17 TIME 1659

INTOXILYZER SERIAL # 80-001455 LOCATION Flagstaff

- 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution, 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0.097 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

02/01/2017

16:59:10

Standard Lot#: 15615100A1

Last Changed By: R. RIVERA #6356

QAS: R. RIVERA #6356

YAVAPAI CSO

Operator: R. RIVERA #6356

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	16:59:48
0.100 Cal Check	0.097	17:00:07
Air Blank	0.000	17:00:36

Preventative Maintenance
Performed -- Timer Reset