

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST R. Rivera AGENCY YAVAPAI CO. SO
DATE 7-9-17 TIME 2338
INTOXILYZER SERIAL # 80-001455 LOCATION Bagdad

- 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0.098 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

17-025679

INTOXILYZER 8000
Location: YAVAPAI SO, BAGDAD
Serial Number: 80-001455
Core Version: 8105.48
07/19/2017 23:38:46

Standard Lot#: 27016100A1
Last Changed By: R. RIVERA #6356

QAS: R. RIVERA #6356
YAVAPAI CSO

Operator: R. RIVERA #6356
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	23:39:16
0.100 Cal Check	0.098	23:39:37
Air Blank	0.000	23:40:07

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST R. Rivera 6356 AGENCY YAVAPAI CO. SO.
DATE 7-19-17 TIME 2340
INTOXILYZER SERIAL # 80-001455 LOCATION BAGDAD

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS


- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0. 000 AC.
- 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS STANDARD CHANGED

SIGNATURE 

11-025679

INTOXILYZER 8000
Location: YAVAPAI SO, BAGDAD
Serial Number: 80-001455
Core Version: 8105.48
07/19/2017

23:40:50

Standard Lot#: 27016100A1
Last Changed By: R. RIVERA #6356

QAS: R. RIVERA #6356
YAVAPAI CSO

Operator: R. RIVERA #6356
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 07/18/1980
Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:41:50
Diagnostic Test	Pass	23:42:25
Air Blank	0.000	23:42:52
0.100 Cal Check	0.098	23:43:13
Air Blank	0.000	23:43:43
Subject Test	0.000	23:44:02
Air Blank	0.000	23:44:30
Wait	ABT*	
Air Blank	IPS**	23:44:45

*Sequence Aborted
**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YAVAPAI SO, BAGDAD
Serial Number: 80-001455
Core Version: 8105.48
07/19/2017 23:53:11

Standard Lot#: 27016100A1
Last Changed By: R. RIVERA #6356

QAS: R. RIVERA #6356
YAVAPAI CSO

Operator: R. RIVERA #6356
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 07/18/1980
Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:54:13
Diagnostic Test	Pass	23:54:48
Air Blank	0.000	23:55:16
0.100 Cal Check	0.098	23:55:37
Air Blank	0.000	23:56:06
Subject Test	0.202*	23:59:14
Air Blank	0.000	23:59:47
Air Blank	0.000	00:00:14
Subject Test	XXX**	00:00:38
Air Blank	RFI***	00:00:46

*Deficient Sample

**Invalid Sample

Begin new deprivation period

***RFI Detect

Not a Successfully Completed Test Sequence

17-025671

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST R. Rivera AGENCY YAVAPAI CO. SO
DATE 7-20-17 TIME 0030
INTOXILYZER SERIAL # 80-001455 LOCATION Bigdad

- 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0.097 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000
Location: YAUAPAI SO, BAGDAD
Serial Number: 80-001455
Core Version: 8105.48
07/20/2017

00:30:41

Standard Lot#: 27016100A1
Last Changed By: R. RIVERA #6356

DAS: R. RIVERA #6356
YAUAPAI CSO

Operator: R. RIVERA #6356
YAUAPAI CSO

Test	g/210L	Time
Air Blank	0.000	00:31:11
0.100 Cal Check	0.097	00:31:32
Air Blank	0.000	00:32:01

Preventative Maintenance
Performed -- Timer Reset

17-025679



ISO/IEC 17025:2005 Accredited Laboratory

Certificate of Analysis

Certificate ID: 9597
Part #: BAC105L100T
Cylinder Size: 105L
Lot Number: 27016100A1
Expiration: 11/5/2018

0.100 BAC (For the calibration of instruments used to determine breath alcohol concentration)

Contents: 105 Liters @ 1000 psig 70°F (21°C)

Component:	Concentration:	Accuracy:	Method:
Ethanol	260 PPM	+/- 0.002 or 2%	NDIR
Nitrogen	BALANCE	BAC whichever is greater	

*NIST Standard Reference Material
 Cylinder No. CC103723 / Job No. 09160202
 Certified 212.8 µmol/mol Ethanol in Nitrogen
 for ILMO Products Co., Jacksonville, IL

Store in dry area, away from sources of heat, ignition and direct sunlight. Do not allow storage area to exceed 52 °C (125 °F).

Jacob Mattes
 Specialty Gas Lab Tech

10/10/16
 Date

Distributed by:

CMI Inc.
 316 East Ninth Street
 Owensboro, KY 42303
 Phone 866-835-0690
www.alcoholtest.com

