

18-029335

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST R. Rivera AGENCY YAVAPAI CO SO
DATE 8-9-18 TIME 0023
INTOXILYZER SERIAL # 80-001455 LOCATION Bagdad

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 097 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

18-029335

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

08/09/2018

00:23:14

Standard Lot#: 27016100A1

Last Changed By: R. RIVERA #6356

QAS: R. RIVERA #6356

YAVAPAI CSO

Operator: R. RIVERA #6356

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	00:23:52
0.100 Cal Check	0.097	00:24:13
Air Blank	0.000	00:24:43

Preventative Maintenance
Performed -- Timer Reset

18-029335

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST R. Rivera 6356 AGENCY YAVAPAI CO SO
DATE 8-9-18 TIME 0026
INTOXILYZER SERIAL # 80-001455 LOCATION BAGLAD

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
() 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
() 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 094 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE  6356

18-029335

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

08/09/2018

00:26:28

Standard Lot#: 27016100A1

Last Changed By: R. RIVERA

#6356

QAS: R. RIVERA

#6356

YAVAPAI CSO

Operator: R. RIVERA

#6356

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 07/18/1980

Sex: M

Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	00:27:48
Diagnostic Test	Pass	00:28:23
Air Blank	0.000	00:28:50
0.100 Cal Check	0.097	00:29:11
Air Blank	0.000	00:29:40
Subject Test	0.000	00:30:09
Air Blank	ABT*	00:30:18
Air Blank	IPS**	00:30:25

*Sequence Aborted

**Improper Sample

Not a Successfully Completed Test Sequence

18-029335

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

08/09/2018

00:32:38

Standard Lot#: 27016100A1

Last Changed By: R. RIVERA

#6356

QAS: R. RIVERA

#6356

YAVAPAI CSO

Operator: R. RIVERA

#6356

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 07/18/1980

Sex: M

Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	00:33:52
Diagnostic Test	Pass	00:34:27
Air Blank	0.000	00:34:54
0.100 Cal Check	0.097	00:35:15
Air Blank	0.000	00:35:45
Subject Test	0.000*	00:38:52
Air Blank	0.000	00:39:26
Air Blank	0.000	00:39:54
Subject Test	XXX**	00:40:26
Air Blank	RFI***	00:40:51

*Deficient Sample

**Invalid Sample

Begin new deprivation period

***RFI Detect

Not a Successfully Completed Test Sequence

18-029335

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST R. Rivera 6356 AGENCY YAVAPAI CO SO
DATE 8-9-18 TIME 0041
INTOXILYZER SERIAL # 80-001455 LOCATION Bagdad

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 094 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  6356

18-029335

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

08/09/2018

00:41:15

Standard Lot#: 27016100A1

Last Changed By: R. RIVERA #6356

QAS: R. RIVERA #6356

YAVAPAI CSO

Operator: R. RIVERA #6356

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	00:41:44
0.100 Cal Check	0.094	00:42:05
Air Blank	0.000	00:42:35

Preventative Maintenance
Performed -- Timer Reset