

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST R. Rivera AGENCY YAVAPAI CSD
 DATE 11-8-25 TIME 2019
 INTOXILYZER SERIAL # 80-001465 LOCATION Bagdad

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
 OR
 Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 097 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

25-053661

INTOXILYZER 8000

Location: YAVAPAI SO, BAGDAD

Serial Number: 80-001455

Core Version: 8105.48

11/08/2025

20:19:37

Standard Lot#: 15924100A1

Last Changed By: R. RIVERA #6356

QAS: R. RIVERA #6356

YAVAPAI CSO

Operator: R. RIVERA #6356

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	20:20:08
0.100 Cal Check	0.097	20:20:29
Air Blank	0.000	20:20:58

Preventative Maintenance
Performed -- Timer Reset