

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Chris Krogedal AGENCY YCSO
DATE 04/06/08 TIME 2245
INTOXILYZER SERIAL # 80-1456 LOCATION Yarnell

- CA 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- CA 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- CA 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- CA 4. Air blank completed.
- CA 5. Calibration check completed. Test results 0. 100 AC.
- CA 6. Air blank completed.
- CA 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Chris Krogedal #570/6535

QAS # 33200

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

04/05/2008

22:40:50

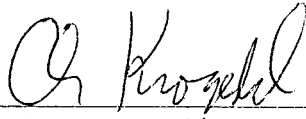
Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535

QAS: C. KROGEDAL #6535

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	22:41:17
Cal Check	0.100	22:41:36
Air Blank	0.000	22:42:05

 #57016535

Operator's Signature

QAS# 33200

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Chris Krogedal AGENCY YCSO
DATE 04/06/08 TIME 2250
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell

(OK) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (OK) 1. Clock time check.
(OK) 2. Date check.

OPERATIONAL TESTS

- (OK) 1. Alcohol-free subject test result 0. 000 AC.
(OK) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(OK) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(OK) 4. Standard Calibration Check standard 0. 101 AC. Result 0. 101 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

RFI Defect not conducted on 4/6/08 due to lack of a radio to
conduct test. Conducted on 04/09/08 at 1652 hrs. successfully.

SIGNATURE

Chris Krogedal #570/6535

QAS #33200

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number : 80-001456

04/05/2008

22:42:57

Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535

QAS: C. KROGEDAL #6535

YAVAPAI CSO

Operator: C. KROGEDAL #6535

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/15/1967

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	22:44:02
Diagnostic Test	Pass	22:44:29
Air Blank	0.000	22:44:56
0.100 Cal Check	0.101	22:45:15
Air Blank	0.000	22:45:45
Subject Test	0.000	22:46:06
Air Blank	ABT*	22:46:14
Air Blank	0.000	22:46:43

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number : 80-001456

04/09/2008

16:52:52

Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535

QAS: C. KROGEDAL #6535

YAVAPAI CSO

Operator: C. KROGEDAL #6535

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/15/1967

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	16:54:02
Air Blank	0.000	16:54:30

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number : 80-001456

04/05/2008

22:46:59

Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535

QAS: C. KROGEDAL #6535

YAVAPAI CSO

Operator: C. KROGEDAL #6535

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/15/1967

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	22:48:10
Diagnostic Test	Pass	22:48:37
Air Blank	IPS*	22:48:49
Air Blank	0.000	22:49:18

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number : 80-001456

04/05/2008

22:49:24

Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535

QAS: C. KROGEDAL #6535

YAVAPAI CSO

Operator: C. KROGEDAL #6535

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/15/1967

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	22:50:11
Diagnostic Test	Pass	22:50:39
Air Blank	0.000	22:51:06
0.100 Cal Check	0.101	22:51:25
Air Blank	0.000	22:51:55
Subject Test	0.074*	22:55:03
Air Blank	0.000	22:55:35
Air Blank	0.000	22:56:02
Subject Test	XXX**	22:56:21
Air Blank	0.000	22:56:56

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence