

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY Yavapai County S.C.  
DATE 7.27.08 TIME 1055  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .1000 AC.  
OR  
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 0.099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Deann Ashby #562 / GAS # 33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

07/27/2008

10:55:59

-----  
Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535  
-----

QAS: D. ASHBY #S62

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	10:56:26
Cal Check	0.099	10:56:45
Air Blank	0.000	10:57:15


  
Operator's Signature #S62/QAS# 33400

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY Yavapai County S.D.  
DATE 7-27-08 TIME 1051/1058  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

(✓) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.  
(✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.  
(✓) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
(✓) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(✓) 4. Standard Calibration Check standard 0. 1000 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

Started before Exhibit G-2. Started over again with G-2  
first. No other problems.

SIGNATURE

Deann Ashby #562 / QAS#33400

INTOXILYZER 8000  
Location: YAVAPAI SO YARNELL  
Serial Number : 80-001456  
07/27/2008

10:51:11

Standard Lot#: 730901I  
Last Changed By: C. KROGEDAL #6535

QAS: D. ASHBY #S62  
YAVAPAI CSO

Operator: D. ASHBY #S62  
YAVAPAI CSO

Subject: SQAP,TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:52:04
Diagnostic Test	Pass	10:52:31
Air Blank	0.000	10:52:58
0.100 Cal Check	0.099	10:53:17
Air Blank	0.000	10:53:47
Subject Test	0.000	10:54:14
Air Blank	0.000	10:54:41
Five Minute Wait	ABT*	
Air Blank	0.000	10:55:18

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YAVAPAI SO YARNELL  
Serial Number : 80-001456  
07/27/2008

10:58:25

Standard Lot#: 730901I  
Last Changed By: C. KROGEDAL #6535

QAS: D. ASHBY #S62  
YAVAPAI CSO

Operator: D. ASHBY #S62  
YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:59:16
Diagnostic Test	Pass	10:59:43
Air Blank	0.000	11:00:10
0.100 Cal Check	0.099	11:00:29
Air Blank	0.000	11:00:59
Subject Test	0.000	11:01:17
Air Blank	ABT*	11:01:25
Air Blank	0.000	11:01:54

\*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number : 80-001456

07/27/2008

11:02:01

Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535

QAS: D. ASHBY #S62

YAVAPAI CSO

Operator: D. ASHBY #S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	11:02:42
Air Blank	0.000	11:03:10

\*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: YAVAPAI SO YARNELL  
Serial Number : 80-001456  
07/27/2008

11:03:29

Standard Lot#: 730901I  
Last Changed By: C. KROGEDAL #6535

QAS: D. ASHBY #S62  
YAVAPAI CSO

Operator: D. ASHBY #S62  
YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:04:21
Diagnostic Test	Pass	11:04:48
Air Blank	0.000	11:05:15
0.100 Cal Check	0.099	11:05:34
Air Blank	0.000	11:06:04
Subject Test	0.020*	11:09:12
Air Blank	0.000	11:09:44
Air Blank	0.000	11:10:11
Subject Test	XXX**	11:10:32
Air Blank	0.000	11:11:06

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**



INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

07/27/2008

11:11:37


-----  
Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535

-----  
QAS: D. ASHBY #S62

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	11:12:04
Cal Check	0.098	11:12:23
Air Blank	0.000	11:12:52

 #S62 / QAS# 33400  
Operator's Signature