

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY Yavapai County S.O.  
DATE 4-20-09 TIME 1045  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.098 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Dawn J Ashby #562 / QAS#33480

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

04/20/2009

10:46:50

-----  
Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535

-----  
QAS: D. ASHBY #S62

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	10:47:18
Cal Check	0.098	10:47:37
Air Blank	0.000	10:48:06

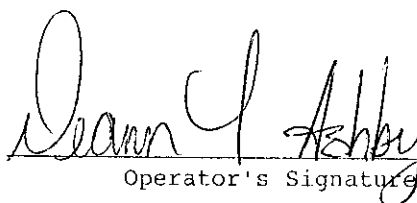
 #S62 / QAS#33400  
Operator's Signature

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. Ashley #5162 AGENCY Yavapai County S.O.  
DATE 4.20.09 TIME ~~1100~~ 1050  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.  
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.  
() 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
() 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

SIGNATURE Dawn Ashley #5162 / QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number : 80-001456

04/20/2009

10:49:11

Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535

QAS: D. ASHBY #S62

YAVAPAI CSO

Operator: D. ASHBY #S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:50:01
Diagnostic Test	Pass	10:50:29
Air Blank	0.000	10:50:56
0.100 Cal Check	0.099	10:51:15
Air Blank	0.000	10:51:44
Subject Test	0.000	10:52:07
Air Blank	0.000	10:52:35
Five Minute Wait	ABT*	
Air Blank	0.000	10:53:42

\*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number : 80-001456

04/20/2009

10:54:13

Standard Lot#: 730901I

Last Changed By: C. KROGEDAL

#6535

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	10:54:59
Air Blank	0.000	10:55:28

\*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number : 80-001456

04/20/2009

10:55:51

Standard Lot#: 730901I

Last Changed By: C. KROGEDAL #6535

QAS: D. ASHBY #S62

YAVAPAI CSO

Operator: D. ASHBY #S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	10:56:27
Air Blank	0.000	10:56:55

\*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: YAVAPAI SO YARNELL  
Serial Number : 80-001456  
04/20/2009 10:57:49

Standard Lot#: 730901I  
Last Changed By: C. KROGEDAL #6535

QAS: D. ASHBY #S62  
YAVAPAI CSO

Operator: D. ASHBY #S62  
YAVAPAI CSO

Subject: SQAP,TEST  
DOB: 01/01/1999  
Sex: M Weight: 150  
15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:58:37
Diagnostic Test	Pass	10:59:05
Air Blank	0.000	10:59:32
0.100 Cal Check	0.098	10:59:51
Air Blank	0.000	11:00:20
Subject Test	0.021*	11:03:29
Air Blank	0.000	11:04:04
Air Blank	0.000	11:04:32
Subject Test	XXX**	11:04:51
Air Blank	0.000	11:05:26

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY Yavapai County S.O.  
DATE 4-20-09 TIME 1105  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(✓) 4. Air blank completed.  
(✓) 5. Calibration check completed. Test results 0. 097 AC.  
(✓) 6. Air blank completed.  
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Deann P Ashby #562 / QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

04/20/2009

11:05:49


-----  
Standard Lot#: 7309011

Last Changed By: C. KROGEDAL #6535  
-----

QAS: D. ASHBY #S62

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	11:06:16
Cal Check	0.097	11:06:35
Air Blank	0.000	11:07:05

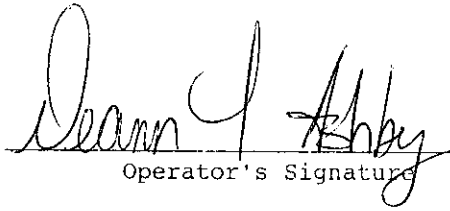 #S62 / QAS #33400  
Operator's Signature