

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY Yavapai County S.O.  
DATE 9-22-09 TIME 0846  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

(✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(✓) 4. Air blank completed.  
(✓) 5. Calibration check completed. Test results 0. 100 AC.  
(✓) 6. Air blank completed.  
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Dawn Ashby #562 QAS#33400

INTOXILYZER 8000  
Location: YAUAPAI CSO YARNELL  
Serial Number: 80-001456  
09/22/2009 08:45:53

-----  
Standard Lot#: 585220  
Last Changed By: D. ASHBY #562  
-----

OAS: D. ASHBY #562  
YAUAPAI CSO  
-----

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | 0.000  | 08:46:20 |
| Cal Check | 0.100  | 08:46:39 |
| Air Blank | 0.000  | 08:47:09 |

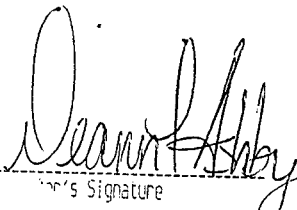
-----  
  
-----  
Operator's Signature

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY Yavapai County 3-0  
DATE 9-22-09 TIME 0851  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.  
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.  
 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 0.99 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE

Deann I. Ashby #562 QAS# 33400

Operate

INTOXILYZER 8000  
Location: YAUAPAI SO YARNELL  
Serial Number: 80-001456  
09/22/2009 08:50:11

Standard Lot#: 585220  
Last Changed By: D. ASHBY #562

QAS: D. ASHBY #562  
YAUAPAI CSO

Operator: D. ASHBY #562  
YAUAPAI CSO

Subject: SOAP, TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

15 Minute Deprivation Period? Yes

| Test            | g/210L | Time     |
|-----------------|--------|----------|
| Air Blank       | 0.000  | 08:51:01 |
| Diagnostic Test | Pass   | 08:51:30 |
| Air Blank       | 0.000  | 08:51:57 |
| 0.100 Cal Check | 0.099  | 08:52:16 |
| Air Blank       | 0.000  | 08:52:45 |
| Subject Test    | 0.000  | 08:53:07 |
| Air Blank       | ABT*   | 08:53:25 |
| Air Blank       | 0.000  | 08:53:54 |

\*Sequence Aborted

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: YAUAPAI SO YARNELL  
Serial Number: 80-001456  
09/22/2009 08:55:34

Standard Lot#: 585220  
Last Changed By: D. ASHBY #562

QAS: D. ASHBY #562  
YAUAPAI CSO

Operator: D. ASHBY #562  
YAUAPAI CSO

Subject: SOAP, TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

15 Minute Deprivation Period? Yes

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | IPS*   | 08:56:14 |
| Air Blank | 0.000  | 08:56:42 |

\*Improper Sample

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: YAUAPAI SO YARNELL  
Serial Number: 80-001456  
09/22/2009 08:57:15

Standard Lot#: 585220  
Last Changed By: D. ASHBY #562

QAS: D. ASHBY #562  
YAUAPAI CSO

Operator: D. ASHBY #562  
YAUAPAI CSO

Subject: SOAP, TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

15 Minute Deprivation Period? Yes

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | RFI*   | 08:57:49 |
| Air Blank | 0.000  | 08:58:17 |

\*RFI Detect

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: YAUAPAI SO YARNELL  
Serial Number: 80-001456  
09/22/2009 08:59:09

-----  
Standard Lot#: 585220  
Last Changed By: D. ASHBY #562  
-----

OAS: D. ASHBY #562  
YAUAPAI CSO  
-----

Operator: D. ASHBY #562  
YAUAPAI CSO  
-----

Subject: SOAP, TEST  
DOB: 01/01/1999  
Sex: M Weight: 150  
-----

15 Minute Deprivation Period? Yes  
-----

| Test            | g/210L | Time     |
|-----------------|--------|----------|
| Air Blank       | 0.000  | 08:59:56 |
| Diagnostic Test | Pass   | 09:00:24 |
| Air Blank       | 0.000  | 09:00:52 |
| 0.100 Cal Check | 0.099  | 09:01:10 |
| Air Blank       | 0.000  | 09:01:40 |
| Subject Test    | 0.197* | 09:04:49 |
| Air Blank       | 0.000  | 09:05:33 |
| Air Blank       | 0.000  | 09:06:00 |
| Subject Test    | XXX**  | 09:06:23 |
| Air Blank       | 0.000  | 09:06:58 |

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period

Not a Successfully  
Completed  
Test Sequence

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY Yavapai County S.O.  
DATE 9.22.09 TIME 0909  
INTOXILYZER SERIAL # 80-901456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.


SIGNATURE Dawn Ashby #562 QAS#33400

INTOXILYZER 8000  
Location: YAUAPAI SO YARNELL  
Serial Number: 80-001456  
09/22/2009 09:09:05

-----  
Standard Lot#: 585220 /  
Last Changed By: D. ASHBY #562  
-----

QAS: D. ASHBY #562  
YAUAPAI CSO  
-----

| Test      | g/210L | Time     |
|-----------|--------|----------|
| Air Blank | 0.000  | 09:09:32 |
| Cal Check | 0.099  | 09:09:51 |
| Air Blank | 0.000  | 09:10:21 |

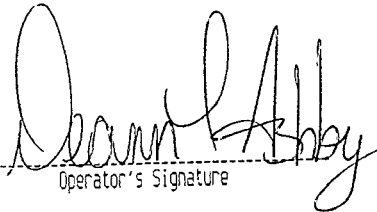  
-----  
Operator's Signature