

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. ASHBY #562 AGENCY YAVAPAI COUNTY SO.

DATE 1.26.10 TIME 0832

INTOXILYZER SERIAL # 80-001456 LOCATION YARNELL SUBSTATION

(✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.098 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Dawn P. Ashby #562 / QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

01/26/2010

08:32:51

-----  
Standard Lot#: 585220

Last Changed By: D. ASHBY #S62

-----  
QAS: D. ASHBY #S62

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	08:33:18
Cal Check	0.098	08:33:37
Air Blank	0.000	08:34:07

 #S62/QAS#33400  
Operator's Signature

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. ASHBY #562 AGENCY YAVAPAI COUNTY SO

DATE 1-26-10 TIME 0834

INTOXILYZER SERIAL # 80-001456 LOCATION YARNELL SUBSTATION

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

() 1. Clock time check.

() 2. Date check.

OPERATIONAL TESTS

() 1. Alcohol-free subject test result 0. 000 AC.

() 2. Error recognition logic system functioning.

Not a Successfully Completed Test Sequence printed

() 3. Proper sample recognition system.

Not a Successfully Completed Test Sequence printed

Deficient sample printed.

() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE D. Ashby #562 / DAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number : 80-001456

01/26/2010

08:34:13

Standard Lot#: 585220

Last Changed By: D. ASHBY

#S62

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:35:03
Diagnostic Test	Pass	08:35:30
Air Blank	0.000	08:35:58
0.100 Cal Check	0.098	08:36:16
Air Blank	0.000	08:36:46
Subject Test	0.000	08:37:07
Air Blank	0.000	08:37:35
Five Minute Wait	ABT*	
Air Blank	0.000	08:38:10

\*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number : 80-001456

01/26/2010

08:38:19

Standard Lot#: 585220

Last Changed By: D. ASHBY

#S62

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	08:38:48
Air Blank	0.000	08:39:17

\*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number : 80-001456

01/26/2010

08:39:22

Standard Lot#: 585220

Last Changed By: D. ASHBY

#S62

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	08:39:51
Air Blank	0.000	08:40:19

\*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number : 80-001456

01/26/2010

08:40:23

Standard Lot#: 585220

Last Changed By: D. ASHBY

#S62

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:41:10
Diagnostic Test	Pass	08:41:37
Air Blank	0.000	08:42:05
0.100 Cal Check	0.098	08:42:23
Air Blank	0.000	08:42:53
Subject Test	0.081*	08:46:01
Air Blank	0.000	08:46:36
Air Blank	0.000	08:47:04
Subject Test	XXX**	08:47:41
Air Blank	0.000	08:48:15

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. ASHBY #562 AGENCY YAVAPAI COUNTY SO.

DATE 1-26-10 TIME 0849

INTOXILYZER SERIAL # 80-001456 LOCATION YARNELL SUBSTATION

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.098 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Dawn Ashby #562 / QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

01/26/2010

08:49:18

-----  
Standard Lot#: 585220

Last Changed By: D. ASHBY #S62

-----  
QAS: D. ASHBY #S62

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	08:49:45
Cal Check	0.098	08:50:04
Air Blank	0.000	08:50:34

 #S62 / QAS # 33400  
Operator's Signature