

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO

DATE 9-21-11 TIME 1310

INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 097 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE D. Ashby #562 / QAS# 33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

09/21/2011

13:10:59

-----  
Standard Lot#: 29510100A1

Last Changed By: D. ASHBY #S62  
-----

QAS: D. ASHBY #S62

YAVAPAI CSO  
-----

Operator: D. ASHBY #S62

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	13:11:31
0.100 Cal Check	0.097	13:11:52
Air Blank	0.000	13:12:21

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO  
DATE 9.21.11 TIME 1315  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

(✓) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

(✓) 1. Clock time check.

(✓) 2. Date check.

OPERATIONAL TESTS

(✓) 1. Alcohol-free subject test result 0. 000 AC.

(✓) 2. Error recognition logic system functioning.

Not a Successfully Completed Test Sequence printed

(✓) 3. Proper sample recognition system.

Not a Successfully Completed Test Sequence printed

Deficient sample printed.

(✓) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES X NO     

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
SIGNATURE Dennis P. Ashby #562 / WAS# 33400

INTOXILYZER 8000  
Location: YAVAPAI SO YARNELL  
Serial Number: 80-001456  
Core Version: 8105.48  
09/21/2011

13:15:57

---

Standard Lot#: 29510100A1  
Last Changed By: D. ASHBY #S62

---

QAS: D. ASHBY #S62  
YAVAPAI CSO

---

Operator: D. ASHBY #S62  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	13:17:06
Diagnostic Test	Pass	13:17:40
Air Blank	0.000	13:18:08
0.100 Cal Check	0.098	13:18:29
Air Blank	0.000	13:18:58
Subject Test	0.000	13:19:15
Air Blank	0.000	13:19:43
Wait	ABT*	
Air Blank	0.000	13:20:21

\*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

09/21/2011

13:21:13

---

Standard Lot#: 29510100A1

Last Changed By: D. ASHBY

#S62

---

QAS: D. ASHBY

#S62

YAVAPAI CSO

---

Operator: D. ASHBY

#S62

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	13:21:56
Air Blank	0.000	13:22:24

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

09/21/2011

13:22:28

---

Standard Lot#: 29510100A1

Last Changed By: D. ASHBY

#S62

---

QAS: D. ASHBY

#S62

YAVAPAI CSO

---

Operator: D. ASHBY

#S62

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	13:23:12
Air Blank	0.000	13:23:41

\*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: YAVAPAI SO YARNELL  
Serial Number: 80-001456  
Core Version: 8105.48  
09/21/2011 13:23:56

---

Standard Lot#: 29510100A1  
Last Changed By: D. ASHBY #S62

---

QAS: D. ASHBY #S62  
YAVAPAI CSO

---

Operator: D. ASHBY #S62  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	13:24:52
Diagnostic Test	Pass	13:25:27
Air Blank	0.000	13:25:54
0.100 Cal Check	0.097	13:26:15
Air Blank	0.000	13:26:45
Subject Test	0.045*	13:29:52
Air Blank	0.000	13:30:24
Air Blank	0.000	13:30:52
Subject Test	XXX**	13:31:15
Air Blank	0.000	13:31:49

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO  
DATE 9-21-11 TIME 1333  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution \_\_\_\_\_ AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 017 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE D. Ashby #562 / QAS # 33400

INTOXILYZER 8000  
Location: YAVAPAI SO YARNELL  
Serial Number: 80-001456  
Core Version: 8105.48  
09/21/2011 13:33:02

-----  
Standard Lot#: 29510100A1  
Last Changed By: D. ASHBY #S62  
-----

QAS: D. ASHBY #S62  
YAVAPAI CSO  
-----

Operator: D. ASHBY #S62  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	13:33:32
0.100 Cal Check	0.097	13:33:53
Air Blank	0.000	13:34:23

Preventative Maintenance  
Performed -- Timer Reset