

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #362 AGENCY YCSO  
DATE 12-18-11 TIME 0658  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 096 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE D. Ashby #362 / GAS# 33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

12/18/2011

06:58:16

-----  
Standard Lot#: 29510100A1

Last Changed By: D. ASHBY #S62  
-----

QAS: D. ASHBY #S62

YAVAPAI CSO  
-----

Operator: D. ASHBY #S62

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	06:59:27
0.100 Cal Check	0.096	06:59:48
Air Blank	0.000	07:00:17

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO  
DATE 12.18.11 TIME 0701  
INTOXILYZER SERIAL # 80-00456 LOCATION Yarnell Substation

(  ) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.
- (  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.
- (  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (  ) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 096 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Deann J Ashby #562 / QAS#33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

12/18/2011

07:01:59

---

Standard Lot#: 29510100A1

Last Changed By: D. ASHBY

#S62

---

QAS: D. ASHBY

#S62

YAVAPAI CSO

---

Operator: D. ASHBY

#S62

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	07:03:28
Diagnostic Test	Pass	07:04:02
Air Blank	0.000	07:04:29
0.100 Cal Check	0.097	07:04:50
Air Blank	0.000	07:05:19
Subject Test	0.000	07:05:37
Air Blank	0.000	07:06:04
Wait	ABT*	
Air Blank	0.000	07:06:41

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

12/18/2011

07:06:48

---

Standard Lot#: 29510100A1

Last Changed By: D. ASHBY

#S62

---

QAS: D. ASHBY

#S62

YAVAPAI CSO

---

Operator: D. ASHBY

#S62

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	07:08:03
Air Blank	0.000	07:08:32

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

12/18/2011

07:09:35

---

Standard Lot#: 29510100A1

Last Changed By: D. ASHBY

#S62

---

OAS: D. ASHBY

#S62

YAVAPAI CSO

---

Operator: D. ASHBY

#S62

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	07:10:10
Air Blank	0.000	07:10:38

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YAVAPAI SO YARNELL  
Serial Number: 80-001456  
Core Version: 8105.48  
12/18/2011

07:12:10

---

Standard Lot#: 29510100A1  
Last Changed By: D. ASHBY #S62

---

QAS: D. ASHBY #S62  
YAVAPAI CSO

---

Operator: D. ASHBY #S62  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	07:12:58
Diagnostic Test	Pass	07:13:32
Air Blank	0.000	07:13:59
0.100 Cal Check	0.096	07:14:20
Air Blank	0.000	07:14:50
Subject Test	0.078*	07:17:57
Air Blank	0.000	07:18:30
Air Blank	0.000	07:18:57
Subject Test	XXX**	07:19:18
Air Blank	0.000	07:19:51

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO  
DATE 12-18-11 TIME 07  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.096 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Deann J Ashby #562/QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

12/18/2011

07:20:59

-----  
Standard Lot#: 29510100A1

Last Changed By: D. ASHBY #S62  
-----

QAS: D. ASHBY #S62

YAVAPAI CSO  
-----

Operator: D. ASHBY #S62

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	07:21:29
0.100 Cal Check	0.096	07:21:50
Air Blank	0.000	07:22:20

Preventative Maintenance  
Performed -- Timer Reset