

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO
DATE 4-16-12 TIME 0855 (0755)
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0: 0.06 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Deann J Ashby #562 / QAS#33400

DPS Form Exh G-2 (Rev 05-01)

Reset clock on machine

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

04/16/2012

08:55:56

Standard Lot#: 29510100A1

Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62

YAVAPAI CSO

Operator: D. ASHBY #S62

YAVAPAI CSO

Test g/210L Time

Air Blank 0.000 08:56:58
0.100 Cal Check 0.096 08:57:19
Air Blank 0.000 08:57:48

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO

DATE 4-16-12 TIME 0800

INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0. 100 AC. Result 0. 095 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS Reset clock on machine

SIGNATURE Dennis Ashby

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

04/16/2012

09:00:06

Standard Lot#: 29510100A1

Last Changed By: D. ASHBY #S62

OAS: D. ASHBY #S62

YAVAPAI CSO

Operator: D. ASHBY #S62

YAVAPAI CSO

Subject: SOAP,TEST

DOB: 01/01/1999

Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:00:51
Diagnostic Test	Pass	09:01:25
Air Blank	0.000	09:01:53
0.100 Cal Check	0.095	09:02:14
Air Blank	0.000	09:02:43
Subject Test	0.000	09:03:01
Air Blank	0.000	09:03:28
Wait	ABT*	
Air Blank	0.000	09:04:04

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YAVAPAI SO YARNELL
Serial Number: 80-001456
Core Version: 8105.48
04/16/2012

08:05:30

Standard Lot#: 29510100A1
Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62
YAVAPAI CSO

Operator: D. ASHBY #S62
YAVAPAI CSO

Subject: SQAP, TEST
DOB: 01/01/1999
Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	08:06:05
Air Blank	0.000	08:06:33

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

04/16/2012

08:06:47

Standard Lot#: 29510100A1

Last Changed By: D. ASHBY

#S62

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	08:07:22
Air Blank	0.000	08:07:50

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YAVAPAI SO YARNELL
Serial Number: 80-001456
Core Version: 8105.48
04/16/2012

08:08:28

Standard Lot#: 29510100A1
Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62
YAVAPAI CSO

Operator: D. ASHBY #S62
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 01/01/1999
Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:09:13
Diagnostic Test	Pass	08:09:47
Air Blank	0.000	08:10:15
0.100 Cal Check	0.095	08:10:36
Air Blank	0.000	08:11:05
Subject Test	0.081*	08:14:13
Air Blank	0.000	08:14:45
Air Blank	0.000	08:15:12
Subject Test	XXX**	08:15:41
Air Blank	0.000	08:16:15

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence


EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO
DATE 4.16.12 TIME 0817
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 095 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Deann G Ashby #562 / GAS #33400

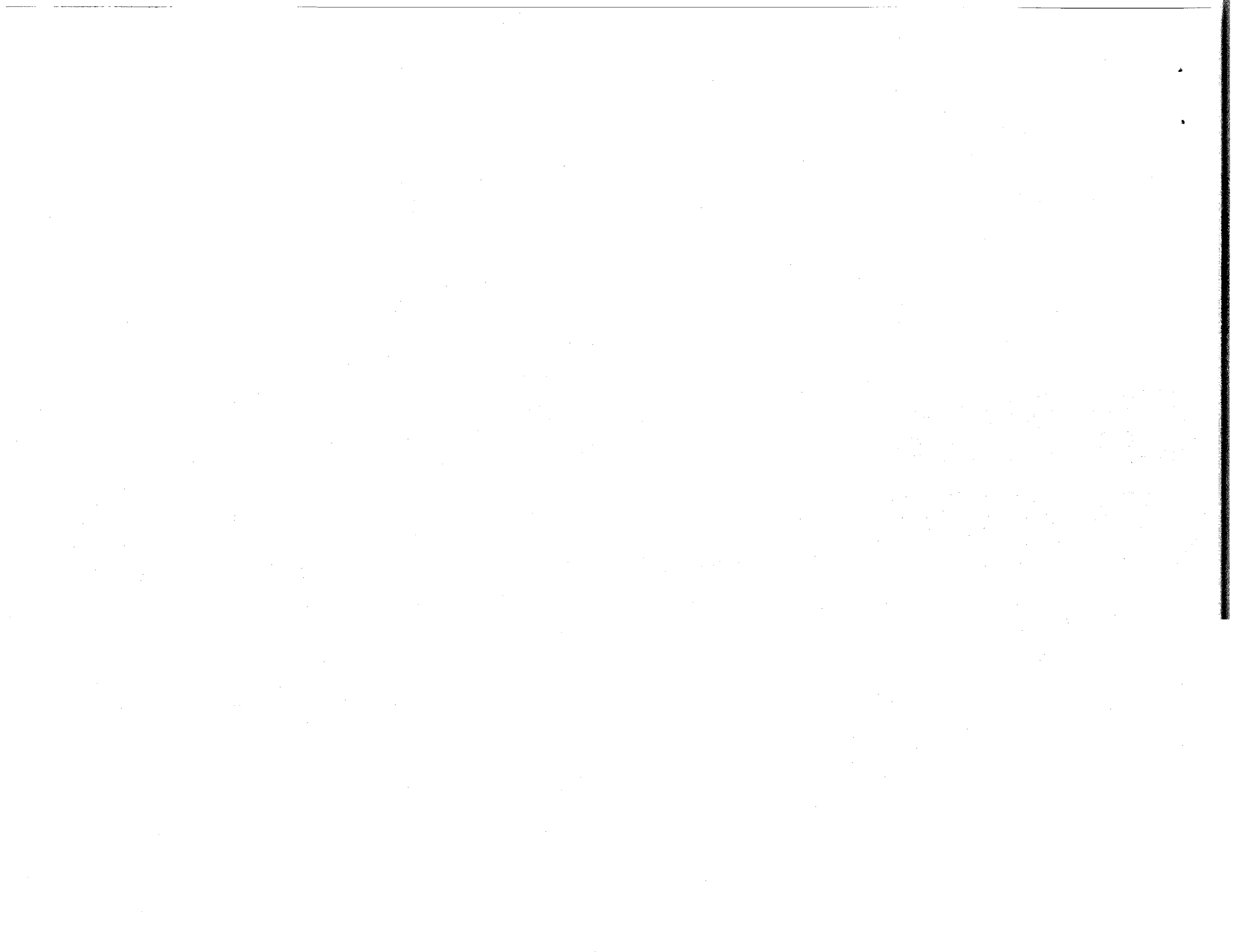

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

04/16/2012

08:17:39

Standard Lot#: 29510100A1

Last Changed By: D. ASHBY #S62

QAS: D. ASHBY #S62

YAVAPAI CSO

Operator: D. ASHBY #S62

YAVAPAI CSO

Test g/210L Time

Air Blank 0.000 08:18:09
0.100 Cal Check 0.095 08:18:30
Air Blank 0.000 08:18:59

Preventative Maintenance
Performed -- Timer Reset