

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YESO  
DATE 2-15-13 TIME 1312  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.098 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Dawn Ashby #562 / QAS # 33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

02/15/2013

13:12:51

-----  
Standard Lot#: 13712100A2

Last Changed By: D. ASHBY #S62

-----  
QAS: D. ASHBY #S62

YAVAPAI CSO

-----  
Operator: D. ASHBY #S62

YAVAPAI CSO

-----  
Test                      g/210L                      Time  
-----  
Air Blank                      0.000                      13:13:26  
0.100 Cal Check                0.098                      13:13:46  
Air Blank                      0.000                      13:14:16

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D Ashby #562 AGENCY YCSO  
DATE 2-15-13 TIME 1315  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.  
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.  
() 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
() 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Deann Ashby #562 / QAS #33400

INTOXILYZER 8000  
Location: YAVAPAI SO YARNELL  
Serial Number: 80-001456  
Core Version: 8105.48  
02/15/2013 13:15:50

---

Standard Lot#: 13712100A2  
Last Changed By: D. ASHBY #S62

---

QAS: D. ASHBY #S62  
YAVAPAI CSO

---

Operator: D. ASHBY #S62  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	13:16:34
Diagnostic Test	Pass	13:17:09
Air Blank	0.000	13:17:36
0.100 Cal Check	0.098	13:17:57
Air Blank	0.000	13:18:27
Subject Test	0.000	13:18:48
Air Blank	0.000	13:19:16
Wait	ABT*	
Air Blank	0.000	13:19:50

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

02/15/2013

13:19:54

Standard Lot#: 13712100A2

Last Changed By: D. ASHBY

#S62

QAS: D. ASHBY

#S62

YAVAPAI CSO

Operator: D. ASHBY

#S62

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	13:20:22
Air Blank	0.000	13:20:50

\*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

02/15/2013

13:20:59

---

Standard Lot#: 13712100A2

Last Changed By: D. ASHBY

#S62

---

QAS: D. ASHBY

#S62

YAVAPAI CSO

---

Operator: D. ASHBY

#S62

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	13:22:32
Air Blank	0.000	13:23:00

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

02/15/2013

13:23:04

---

Standard Lot#: 13712100A2

Last Changed By: D. ASHBY

#S62

---

QAS: D. ASHBY

#S62

YAVAPAI CSO

---

Operator: D. ASHBY

#S62

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	13:23:48
Diagnostic Test	Pass	13:24:23
Air Blank	0.000	13:24:50
0.100 Cal Check	0.098	13:25:11
Air Blank	0.000	13:25:41
Subject Test	0.075*	13:28:48
Air Blank	0.000	13:29:23
Air Blank	0.000	13:29:49
Subject Test	XXX**	13:30:07
Air Blank	0.000	13:30:41

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #562 AGENCY YCSO  
DATE 2-15-13 TIME 1330  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 097 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Dawn Ashby #562/QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

02/15/2013

13:30:53

-----  
Standard Lot#: 13712100A2

Last Changed By: D. ASHBY #S62

-----  
QAS: D. ASHBY #S62

YAVAPAI CSO

-----  
Operator: D. ASHBY #S62

YAVAPAI CSO

-----  
Test                      g/210L                      Time  
-----  
Air Blank                      0.000                      13:31:23  
0.100 Cal Check                0.097                      13:31:44  
Air Blank                      0.000                      13:32:14

Preventative Maintenance  
Performed -- Timer Reset