

# COPY

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #2271 AGENCY YCSO

DATE 11.19.13 TIME 0932

INTOXILYZER SERIAL # 80.001456 LOCATION Yarnell Substation

(✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR

Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$

- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"  
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.  
(✓) 4. Air blank completed.  
(✓) 5. Calibration check completed. Test results 0. 096 AC.  
(✓) 6. Air blank completed.  
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE D. Ashby #2271 / QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

11/19/2013

09:52:43

-----  
Standard Lot#: 13712100A2

Last Changed By: D. ASHBY #S62  
-----

QAS: D. ASHBY #2271

YAVAPAI CSO  
-----

Operator: D. ASHBY #2271

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	09:53:54
0.100 Cal Check	0.096	09:53:56
Air Blank	0.000	09:54:05

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. Ashby #2271 AGENCY YCSO  
DATE 11.19.13 TIME 0935  
INTOXILYZER SERIAL # 80.001456 LOCATION Yarnell Substation

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.  
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.  
 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 096 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Dean Ashby #2271 / QAS #33400

INTOXILYZER 8000  
Location: YAVAPAI SO YARNELL  
Serial Number: 80-001456  
Core Version: 8105.48  
11/19/2013

09:35:07

---

Standard Lot#: 13712100A2  
Last Changed By: D. ASHBY #862

---

QAS: D. ASHBY #2271  
YAVAPAI CSO

---

Operator: D. ASHBY #2271  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	09:35:53
Diagnostic Test	Pass	09:36:35
Air Blank	0.000	09:36:55
0.100 Cal Check	0.096	09:37:16
Air Blank	0.000	09:37:41
Subject Test	0.000	09:38:05
Air Blank	0.000	09:38:32
Wait	ABT*	
Air Blank	0.000	09:39:32

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

11/19/2013

09:39:40

---

Standard Lot#: 13712100A2

Last Changed By: D. ASHBY

#562

---

QAS: D. ASHBY

#2271

YAVAPAI CSO

---

Operator: D. ASHBY

#2271

YAVAPAI CSO

---

Subject: SOAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	09:40:08
Air Blank	0.000	09:40:36

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YAVAPAI SO YARNELL  
Serial Number: 80-001456  
Core Version: 8105.48  
11/19/2013

09:40:43

---

Standard Lot#: 13712100A2  
Last Changed By: D. ASHBY #862

---

QAS: D. ASHBY #2271  
YAVAPAI CSO

---

Operator: D. ASHBY #2271  
YAVAPAI CSO

---

Subject: SOAP,TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	09:41:34
Air Blank	0.000	09:42:02

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YAVAPAI SO YARNELL  
Serial Number: 80-001456  
Core Version: 8105.48  
11/19/2013

09:42:22

---

Standard Lot#: 13712100A2  
Last Changed By: D. ASHBY #862

---

QAS: D. ASHBY #2271  
YAVAPAI CSO

---

Operator: D. ASHBY #2271  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	09:43:44
Diagnostic Test	Pass	09:44:19
Air Blank	0.000	09:44:46
0.100 Cal Check	0.096	09:45:07
Air Blank	0.000	09:45:37
Subject Test	0.094*	09:48:44
Air Blank	0.000	09:49:17
Air Blank	0.000	09:49:44
Subject Test	XXX**	09:50:13
Air Blank	0.000	09:50:46

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby # 2271 AGENCY YCSO

DATE 11.19.13 TIME 0951

INTOXILYZER SERIAL # 80.001456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 096 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Deann Ashby # 2271 / DAS# 33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

11/19/2013

09:51:40

-----  
Standard Lot#: 13712100A2

Last Changed By: D. ASHBY #S62  
-----

QAS: D. ASHBY #2271

YAVAPAI CSO  
-----

Operator: D. ASHBY #2271

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	09:53:13
0.100 Cal Check	0.096	09:52:57
Air Blank	0.000	09:53:01

Preventative Maintenance  
Performed -- Timer Reset