

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #2271 AGENCY YCSO  
DATE 6-16-14 TIME 1201  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 095 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Dawn Ashby #2271 / QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

06/16/2014

12:01:30

-----  
Standard Lot#: 13712100A2

Last Changed By: D. ASHBY #S62  
-----

QAS: D. ASHBY #2271

YAVAPAI CSO  
-----

Operator: D. ASHBY #2271

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	12:02:11
0.100 Cal Check	0.095	12:02:32
Air Blank	0.000	12:03:02
Air Blank	0.000	12:05:30

\*Input Timed Out

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D Ashby #2271 AGENCY YCSO  
DATE 6.16.14 TIME 1211  
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

(✓) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.  
(✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.  
(✓) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
(✓) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(✓) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 095 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE

Dawn Ashby #2271 / QAS# 33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

06/16/2014

12:11:32

---

Standard Lot#: 13712100A2

Last Changed By: D. ASHBY

#S62

---

QAS: D. ASHBY

#2271

YAVAPAI CSO

---

Operator: D. ASHBY

#2271

YAVAPAI CSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	12:13:15
Diagnostic Test	Pass	12:13:51
Air Blank	0.000	12:14:18
0.100 Cal Check	0.096	12:14:39
Air Blank	0.000	12:15:09
Subject Test	0.000	12:15:29
Air Blank	0.000	12:15:56
Wait	ABT*	
Air Blank	0.000	12:16:54

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YAVAPAI SO YARNELL  
Serial Number: 80-001456  
Core Version: 8105.48  
06/16/2014 12:17:02

---

Standard Lot#: 13712100A2  
Last Changed By: D. ASHBY #S62

---

QAS: D. ASHBY #2271  
YAVAPAI CSO

---

Operator: D. ASHBY #2271  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	12:18:19
Air Blank	0.000	12:18:47

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

06/16/2014

12:29:12

---

Standard Lot#: 13712100A2

Last Changed By: D. ASHBY

#S62

---

QAS: D. ASHBY

#2271

YAVAPAI CSO

---

Operator: D. ASHBY

#S62

YCSO

---

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	12:30:07
Air Blank	0.000	12:30:36

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YAVAPAI SO YARNELL  
Serial Number: 80-001456  
Core Version: 8105.48  
06/16/2014 12:30:40

---

Standard Lot#: 13712100A2  
Last Changed By: D. ASHBY #S62

---

QAS: D. ASHBY #2271  
YAVAPAI CSO

---

Operator: D. ASHBY #S62  
YCSO

---

Subject: SQAP,TEST  
DOB: 01/01/1999  
Sex: M Weight: 150

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	12:31:42
Diagnostic Test	Pass	12:32:17
Air Blank	0.000	12:32:44
0.100 Cal Check	0.095	12:33:05
Air Blank	0.000	12:33:35
Subject Test	0.083*	12:36:42
Air Blank	0.000	12:37:15
Air Blank	0.000	12:37:42
Subject Test	XXX**	12:38:41
Air Blank	0.000	12:39:17

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D Ashley #2271 AGENCY YCSO  
DATE 6.16.14 TIME 1239  
INTOXILYZER SERIAL # 80.001456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 095 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Dawn Ashley #2271 / QAS #33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

06/16/2014

12:39:46

-----  
Standard Lot#: 13712100A2

Last Changed By: D. ASHBY #S62  
-----

QAS: D. ASHBY #2271

YAVAPAI CSO  
-----

Operator: D. ASHBY #2271

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	12:40:16
0.100 Cal Check	0.095	12:40:36
Air Blank	0.000	12:41:06

Preventative Maintenance

Performed -- Timer Reset