

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #2271 AGENCY YCSO
DATE 9-10-14 TIME 0752
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration check completed. Test results 0. 099 AC.
(✓) 6. Air blank completed.
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Deann Ashby #2271 / QAS # 33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

09/10/2014

07:52:54

Standard Lot#: 15013100A2

Last Changed By: D. ASHBY #2271

QAS: D. ASHBY #2271

YAVAPAI CSO

Operator: D. ASHBY #2271

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	07:53:27
0.100 Cal Check	0.099	07:53:48
Air Blank	0.000	07:54:18

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST D. Ashby #2271 AGENCY YCSO
DATE 9.10.14 TIME 0754
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell Substation

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
() 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
() 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Denn Ashby #2271 / QAS # 33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

09/10/2014

07:54:35

Standard Lot#: 15013100A2

Last Changed By: D. ASHBY #2271

QAS: D. ASHBY #2271

YAVAPAI CSO

Operator: D. ASHBY #2271

YAVAPAI CSO

Subject: SOAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:55:52
Diagnostic Test	Pass	07:56:27
Air Blank	0.000	07:56:54
0.100 Cal Check	0.099	07:57:15
Air Blank	0.000	07:57:45
Subject Test	0.000	07:58:05
Air Blank	0.000	07:58:32
Wait	ABT*	
Air Blank	0.000	07:59:09

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

09/10/2014

07:59:13

Standard Lot#: 15013100A2

Last Changed By: D. ASHBY

#2271

QAS: D. ASHBY

#2271

YAVAPAI CSO

Operator: D. ASHBY

#2271

YAVAPAI CSO

Subject: SOAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	08:00:35
Air Blank	0.000	08:01:03

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

09/10/2014

08:01:38

Standard Lot#: 15013100A2

Last Changed By: D. ASHBY

#2271

QAS: D. ASHBY

#2271

YAVAPAI CSO

Operator: D. ASHBY

#2271

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 01/01/1999

Sex: M

Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	08:02:35
Air Blank	0.000	08:03:04

*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YAVAPAI SO YARNELL
Serial Number: 80-001456
Core Version: 8105.48
09/10/2014 08:03:22

Standard Lot#: 15013100A2
Last Changed By: D. ASHBY #2271

QAS: D. ASHBY #2271
YAVAPAI CSO

Operator: D. ASHBY #2271
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 01/01/1999
Sex: M Weight: 150

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	08:04:10
Diagnostic Test	Pass	08:04:45
Air Blank	0.000	08:05:13
0.100 Cal Check	0.099	08:05:33
Air Blank	0.000	08:06:03
Subject Test	0.053*	08:09:11
Air Blank	0.000	08:09:44
Air Blank	0.000	08:10:11
Subject Test	XXX**	08:10:36
Air Blank	0.000	08:11:12

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST D. Ashby #2271 AGENCY YCSO
DATE 9.10.14 TIME 0811
INTOXILYZER SERIAL # 80.001456 LOCATION Yarnell Substation

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(✓) 4. Air blank completed.
(✓) 5. Calibration Check completed. Test results 0. 097 AC.
(✓) 6. Air blank completed.
(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Deann Ashby #2271 / QAS # 33400

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

09/10/2014

08:11:35

Standard Lot#: 15013100A2

Last Changed By: D. ASHBY #2271

QAS: D. ASHBY #2271

YAVAPAI CSO

Operator: D. ASHBY #2271

YAVAPAI CSO

Test g/210L Time

Air Blank 0.000 08:12:06
0.100 Cal Check 0.097 08:12:26
Air Blank 0.000 08:12:56

Preventative Maintenance
Performed -- Timer Reset