

COPY

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Murray AGENCY Yavapai County Sheriff's Office
DATE 1/13/2015 TIME 13:44:41
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.098 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

01/13/2015

13:44:43

Standard Lot#: 15013100A2

Last Changed By: D. ASHBY #2271

QAS: J. MURRAY5597 #N56

YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56

YAVAPAI SHERIFFS

Test	g/210L	Time
Air Blank	0.000	13:45:26
0.100 Cal Check	0.098	13:45:47
Air Blank	0.000	13:46:16

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. Murray AGENCY Yavapai County Sheriff's Office
DATE 1/13/2015 TIME 13:48:38
INTOXILYZER SERIAL # 80-001456 LOCATION Yarnell

(X) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (X) 1. Clock time check.
(X) 2. Date check.

OPERATIONAL TESTS

- (X) 1. Alcohol-free subject test result 0. 000 AC.
(X) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
(X) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
(X) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES X NO _____

COMMENTS 15013100A2 6/1/2015 +900 PSI

SIGNATURE J. Murray NS6/5597

INTOXILYZER 8000
Location: YAVAPAI SO YARNELL
Serial Number: 80-001456
Core Version: 8105.48
01/13/2015

13:48:38

Standard Lot#: 15013100A2
Last Changed By: D. ASHBY #2271

OAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Subject: SOAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:49:32
Diagnostic Test	Pass	13:50:07
Air Blank	0.000	13:50:34
0.100 Cal Check	0.098	13:50:55
Air Blank	0.000	13:51:25
Subject Test	0.000	13:51:49
Air Blank	0.000	13:52:16
Wait	ABT*	
Air Blank	RFI**	13:52:31

*Sequence Aborted

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

01/13/2015

13:52:35

Standard Lot#: 15013100A2

Last Changed By: D. ASHBY

#2271

QAS: J. MURRAY5597

#N56

YAVAPAI SHERIFFS

Operator: J. MURRAY5597

#N56

YAVAPAI SHERIFFS

Subject: SQAP,TEST

DOB: 10/20/1960

Sex: F

Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	13:53:19
Air Blank	0.000	13:53:47

*Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YAVAPAI SO YARNELL
Serial Number: 80-001456
Core Version: 8105.48
01/13/2015

13:54:11

Standard Lot#: 15013100A2
Last Changed By: D. ASHBY #2271

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Subject: SQAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:55:11
Diagnostic Test	Pass	13:55:46
Air Blank	0.000	13:56:13
0.100 Cal Check	0.097	13:56:34
Air Blank	0.000	13:57:04
Subject Test	0.110*	14:00:12
Air Blank	0.000	14:00:45
Air Blank	0.000	14:01:13
Subject Test	RGE**	14:01:44
Air Blank	0.000	14:02:21

*Deficient Sample

**Range Exceeded

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

01/13/2015

14:02:47

Standard Lot#: 15013100A2

Last Changed By: D. ASHBY

#2271

QAS: J. MURRAY5597

#N56

YAVAPAI SHERIFFS

Operator: J. MURRAY5597

#N56

YAVAPAI SHERIFFS

Subject: SQAP,TEST

DOB: 10/20/1960

Sex: F

Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:03:40
Diagnostic Test	Pass	14:04:15
Air Blank	0.000	14:04:42
0.100 Cal Check	0.097	14:05:03
Air Blank	0.000	14:05:33
Subject Test	XXX*	14:06:01
Air Blank	0.000	14:06:36

*Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Murray NS6/5597 AGENCY Yavapai County Sheriff's Office
DATE 1/13/2015 TIME 14:06:52
INTOXILYZER SERIAL # 80-00 1456 LOCATION Yarnell

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.097 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray NS6/5597

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

01/13/2015

14:06:52

Standard Lot#: 15013100A2

Last Changed By: D. ASHBY #2271

QAS: J. MURRAY5597 #N56

YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56

YAVAPAI SHERIFFS

Test	g/210L	Time
Air Blank	0.000	14:07:42
0.100 Cal Check	0.097	14:08:03
Air Blank	0.000	14:08:32

Preventative Maintenance
Performed -- Timer Reset