



ARIZONA DEPARTMENT OF PUBLIC SAFETY

INTOXILYZER MAINTENANCE REPORT

CR NO. I800014560005

Page 1 of 1

AGENCY Yavapai County Sheriff's Office
Prescott, AZ 86301
OFFICER ASHBY, #2271
DATE April 28, 2016



INSTRUMENT

80001456

INSTRUMENT LOCATION

Yavapai County Sheriff's Office

MAINTENANCE REQUESTED

Preventative Maintenance

MAINTENANCE PERFORMED

Preventative Maintenance Performed:

Inspection, Calibration, Verification

A handwritten signature in black ink, appearing to read "Robert W. Stephenson".

ROBERT W. STEPHENSON, #5742, Forensic Scientist
Central Regional Crime Laboratory
2102 W. Encanto Blvd., Phoenix, AZ 85009
602-223-2114

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST R. Rivera AGENCY YAVAPAI County S.O
DATE 5-4-16 TIME 1812
INTOXILYZER SERIAL # 80-001456 LOCATION YARNELL

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 098 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

05/04/2016

18:12:14

Standard Lot#: 36314100A4

Last Changed By: D. ASHBY #2271

QAS: R. RIVERA #6356

YAVAPAI CSO

Operator: R. RIVERA #6356

YAVAPAI CSO

Test g/210L Time

Air Blank 0.000 18:12:45
0.100 Cal Check 0.098 18:13:05
Air Blank 0.000 18:13:35

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST R. RIVERA AGENCY YAVAPAI COUNTY SO
DATE 5-4-16 TIME 1812
INTOXILYZER SERIAL # 80-001456 LOCATION YARNELL

() 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
() 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0. 000 AC.
() 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
() 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
() 4. Standard Calibration Check standard 0. 100 AC. Result 0. _____ AC.

Instrument is operating properly and accurately. YES _____ NO _____

COMMENTS

SIGNATURE [Signature]

INTOXILYZER 8000
Location: YAVAPAI SO YARNELL
Serial Number: 80-001456
Core Version: 8105.48
05/04/2016

18:13:59

Standard Lot#: 36314100A4
Last Changed By: D. ASHBY #2271

QAS: R. RIVERA #6356
YAVAPAI CSO

Operator: R. RIVERA #6356
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 07/18/1980
Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:15:19
Diagnostic Test	Pass	18:15:54
Air Blank	0.000	18:16:22
0.100 Cal Check	0.098	18:16:43
Air Blank	0.000	18:17:13
Subject Test	0.000	18:17:34
Air Blank	ABT*	18:17:42
Air Blank	IPS**	18:17:50

*Sequence Aborted
**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: YAVAPAI SO YARNELL
Serial Number: 80-001456
Core Version: 8105.48
05/04/2016 18:24:16

Standard Lot#: 36314100A4
Last Changed By: D. ASHBY #2271

QAS: R. RIVERA #6356
YAVAPAI CSO

Operator: R. RIVERA #6356
YAVAPAI CSO

Subject: SQAP,TEST
DOB: 07/18/1980
Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	18:26:38
Diagnostic Test	Pass	18:27:12
Air Blank	0.000	18:27:40
0.100 Cal Check	0.097	18:28:01
Air Blank	0.000	18:28:30
Subject Test	0.013*	18:31:38
Air Blank	0.000	18:32:27
Air Blank	0.000	18:32:55
Subject Test	XXX**	18:33:33
Air Blank	RFI***	18:33:44

*Deficient Sample

**Invalid Sample

Begin new deprivation period

***RFI Detect

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST R. Rivera AGENCY YAVAPAI COUNTY S.O
DATE 5-9-16 TIME 1838
INTOXILYZER SERIAL # 80-001456 LOCATION YARNELL

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.096 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Rivera 6356

INTOXILYZER 8000

Location: YAVAPAI SO YARNELL

Serial Number: 80-001456

Core Version: 8105.48

05/04/2016

18:38:09

Standard Lot#: 36314100A4

Last Changed By: D. ASHBY #2271

QAS: R. RIVERA #6356

YAVAPAI CSO

Operator: R. RIVERA #6356

YAVAPAI CSO

Test g/210L Time

Air Blank 0.000 18:38:40
0.100 Cal Check 0.096 18:39:01
Air Blank 0.000 18:39:31

Preventative Maintenance
Performed -- Timer Reset