

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST R. Rivera AGENCY YAVAPAI CO. SO
DATE 6-25-17 TIME 2121
INTOXILYZER SERIAL # 80-001456 LOCATION YARNELL

- 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0. 098 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE 

INTOXILYZER 8000
Instrument Initialization
21:00:05 06/25/2017

INTOXILYZER 8000
Location: YAUAPAI SO YARNELL
Serial Number: 80-001456
Core Version: 8135.48
06/25/2017 21:21:12

Standard Lot#: 27016100A1
Last Changed By: D. ASHBY #2271

GAS: R. RIVERA #6356
YAUAPAI CSO

Operator: R. RIVERA #6356
YAUAPAI CSO

Test	g/210L	Time
Air Blank	0.000	21:21:42
0.100 Cal Check	0.098	21:22:02
Air Blank	0.000	21:22:32

Preventative Maintenance
Performed -- Timer Reset

17-00000

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Rivera AGENCY YAVAPAI CO. SO
DATE 6-25-17 TIME 2131
INTOXILYZER SERIAL # 80-001454 LOCATION YARNELL

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0. 000 AC.
- 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS Instrument was sent in for Annual
Maintenance, Put back into service

SIGNATURE [Signature]

11-022600

INTOXILYZER 8000
Location: YAUAPAI SO YARNELL
Serial Number: 80-001456
Core Version: 8105.48
06/25/2017

21:31:22

Standard Lot#: 27016100A1
Last Changed By: D. ASHBY #2271

OAS: R. RIVERA #6356
YAUAPAI CSO

Operator: R. RIVERA #6356
YAUAPAI CSO

Subject: SQAP, TEST
DOB: 07/18/1980
Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:33:02
Diagnostic Test	Pass	21:33:39
Air Blank	0.000	21:34:07
0.100 Cal Check	0.098	21:34:28
Air Blank	0.000	21:34:58
Subject Test	0.000	21:35:15
Air Blank	0.000	21:35:42
Wait	ABT*	
Air Blank	IPS**	21:35:56

*Sequence Aborted
**Improper Sample

Not a Successfully
Completed
Test Sequence

INTOXILYZER 8000
Location: YAUAPAI SO YARNELL
Serial Number: 80-001456
Core Version: 8105.48
06/25/2017 21:38:37

Standard Lot#: 27016100A1
Last Changed By: D. ASHBY #2271

DAS: R. RIVERA #6356
YAUAPAI CSO

Operator: R. RIVERA #6356
YAUAPAI CSO

Subject: SQAP, TEST
DOB: 07/18/1980
Sex: M Weight: 123

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:39:39
Diagnostic Test	Pass	21:40:17
Air Blank	0.000	21:40:44
0.100 Cal Check	0.099	21:41:05
Air Blank	0.000	21:41:35
Subject Test	0.260*	21:44:42
Air Blank	0.000	21:45:21
Air Blank	0.000	21:45:48
Subject Test	RGE**	21:46:24
Air Blank	RFI***	21:46:36

*Deficient Sample
**Range Exceeded
***RFI Detect

Not a Successfully
Completed
Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST R. Rivera AGENCY Yacapai
DATE 6-25-17 TIME 2149
INTOXILYZER SERIAL # 80-001456 LOCATION YACAPAI

- 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution, 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 4. Air blank completed.
- 5. Calibration check completed. Test results 0. 098 AC.
- 6. Air blank completed.
- 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

INTOXILYZER 8000
Location: YAUAPAI SO YARNELL
Serial Number: 80-001456
Core Version: 8105.48
06/25/2017 21:51:08

Standard Lot#: 27016100A1
Last Changed By: D. ASHBY #2271

QAS: R. RIVERA #6356
YAUAPAI CSO

Operator: R. RIVERA #6356
YAUAPAI CSO

Test	g/210L	Time
Air Blank	0.000	21:51:38
0.100 Cal Check	0.098	21:51:58
Air Blank	0.000	21:52:28

Preventative Maintenance
Performed -- Timer Reset

INTOXILYZER 8000
Location: YAUAPAI SO YARNELL
Serial Number: 80-001456
Core Version: 8105.48
06/25/2017 21:49:51

Standard Lot#: 27016100A1
Last Changed By: D. ASHBY #2271

QAS: R. RIVERA #6356
YAUAPAI CSO

Operator: R. RIVERA #6356
YAUAPAI CSO

Test	g/210L	Time
Air Blank	AMB*	21:50:22
Air Blank	0.000	21:50:50

*Ambient Fail

170600



ARIZONA DEPARTMENT OF PUBLIC SAFETY

INTOXILYZER MAINTENANCE REPORT

CR NO. I800014560006

Page 1 of 1

AGENCY Yavapai County Sheriff's Office
Prescott, AZ 86301
OFFICER RIVERA #6356
DATE June 19, 2017



INSTRUMENT

80001456

INSTRUMENT LOCATION

Yavapai County Sheriff's Office

MAINTENANCE REQUESTED

Preventative Maintenance

MAINTENANCE PERFORMED

Preventative Maintenance Performed:

Inspection, Calibration, Verification

Erin E Boone

ERIN E. BOONE, #5823, Forensic Scientist
Central Regional Crime Laboratory
2102 W. Encanto Blvd., Phoenix, AZ 85009
602-223-2281