

COPY

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Murray NSB AGENCY Yavapai Co. Sheriff's  
DATE 1-5-11 TIME 9:00:05  
INTOXILYZER SERIAL # 80-01734 LOCATION Selma Office

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution        AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray NSB

INTOXILYZER 8000  
Location: YCSO SELIGMAN  
Serial Number: 80-001734  
Core Version: 8105.48  
01/05/2011

09:00:05

-----  
Standard Lot#: 779386  
Last Changed By: J. MURRAY #N56  
-----

QAS: J. MURRAY #N56  
YAVAPAI YCSO  
-----

Operator: J. MURRAY #N56  
YAVAPAI YCSO  
-----

Test	g/210L	Time
Air Blank	0.000	09:00:42
0.100 Cal Check	0.100	09:01:03
Air Blank	0.000	09:01:33

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. MUGRAJ NSB AGENCY Yavapai Co. Sheriff's  
DATE 1-5-11 TIME 9:06:43  
INTOXILYZER SERIAL # 80-01734 LOCATION Seligman Office

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.  
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.  
 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 099 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

779386 - 100  
1000 PSI

SIGNATURE J. MUGRAJ NSB

INTOXILYZER 8000  
Location: YCSO SELIGMAN  
Serial Number: 80-001734  
Core Version: 8105.48  
01/05/2011 09:06:43

---

Standard Lot#: 779386  
Last Changed By: J. MURRAY #N56

---

QAS: J. MURRAY #N56  
YAVAPAI YCSO

---

Operator: J. MURRAY #N56  
YAVAPAI YCSO

---

Subject: SQAP,TEST  
DOB: 10/20/1960  
Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	09:07:51
Diagnostic Test	Pass	09:08:26
Air Blank	0.000	09:08:53
0.100 Cal Check	0.099	09:09:14
Air Blank	0.000	09:09:44
Subject Test	0.000	09:10:19
Air Blank	0.000	09:10:46
Wait	ABT*	
Air Blank	RFI**	09:11:03

\*Sequence Aborted  
\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YCSO SELIGMAN

Serial Number: 80-001734

Core Version: 8105.48

01/05/2011

09:11:32

---

Standard Lot#: 779386

Last Changed By: J. MURRAY

#N56

---

QAS: J. MURRAY

#N56

YAVAPAI YCSO

---

Operator: J. MURRAY

#N56

YAVAPAI YCSO

---

Subject: SQAP,TEST

DOB: 10/20/1960

Sex: F

Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	09:12:19
Air Blank	RFI**	09:12:34

\*Improper Sample

\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YCSO SELIGMAN  
Serial Number: 80-001734  
Core Version: 8105.48  
01/05/2011

09:12:55

---

Standard Lot#: 779386  
Last Changed By: J. MURRAY #N56

---

QAS: J. MURRAY #N56  
YAVAPAI YCSO

---

Operator: J. MURRAY #N56  
YAVAPAI YCSO

---

Subject: SQAP,TEST  
DOB: 10/20/1960  
Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	09:13:53
Diagnostic Test	Pass	09:14:30
Air Blank	0.000	09:14:57
0.100 Cal Check	0.099	09:15:18
Air Blank	0.000	09:15:47
Subject Test	0.152*	09:18:55
Air Blank	0.000	09:19:29
Air Blank	0.000	09:19:56
Subject Test	XXX**	09:20:29
Air Blank	0.000	09:21:05

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST MURRAY NSB AGENCY Yavapai Co. Sheriff's  
DATE 1/5/11 TIME 9:22:16  
INTOXILYZER SERIAL # 80001734 LOCATION Seligman Office

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution        AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0.099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray NSB

INTOXILYZER 8000  
Location: YCSO SELIGMAN  
Serial Number: 80-001734  
Core Version: 8105.48  
01/05/2011

09:22:16

-----  
Standard Lot#: 779386  
Last Changed By: J. MURRAY #N56  
-----

QAS: J. MURRAY #N56  
YAVAPAI YCSO  
-----

Operator: J. MURRAY #N56  
YAVAPAI YCSO  
-----

Test	g/210L	Time
Air Blank	0.000	09:22:52
0.100 Cal Check	0.099	09:23:13
Air Blank	0.000	09:23:42

Preventative Maintenance  
Performed -- Timer Reset