

COPY

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Murray AGENCY Yavapai County Sheriff's Office  
DATE 8/26/2013 TIME 10:16:17  
INTOXILYZER SERIAL # 80-01734 LOCATION Seligman 103

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution          AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 098 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray NS6/5597

DPS Form Exh G-2 (Rev 05-01)

QA SPECIALIST

DATE

TIME

INTOXILYZER SERIAL #

LOCATION

TEST RESULTS

TEST RESULTS

TEST RESULTS

TEST RESULTS

TEST RESULTS

TEST RESULTS

TEST RESULTS

TEST RESULTS

INTOXILYZER 8000

Location: YCSO SELIGMAN

Serial Number: 80-001734

Core Version: 8105.48

05/26/2013

10:16:17

-----  
Standard Lot#: 09312100A1

Last Changed By: J. MURRAY #N56

-----  
QAS: J. MURRAY #N56

YAVAPAI CSO

-----  
Operator: J. MURRAY #N56

YAVAPAI CSO

-----  
Test                      g/210L                      Time  
-----  
Air Blank                      0.000                      10:17:09  
0.100 Cal Check                0.098                      10:17:30  
Air Blank                      0.000                      10:17:59

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. Murry AGENCY Yavapai County Sheriff's Office  
DATE 5/26/2013 TIME 10:21:14  
INTOXILYZER SERIAL # 80-01734 LOCATION Seligman 103

(x) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (x) 1. Clock time check.
- (x) 2. Date check.

OPERATIONAL TESTS

- (x) 1. Alcohol-free subject test result 0. 000 AC.
- (x) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (x) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (x) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

09322100A1  
8504 PSL 6/1/2014  
Display

TESTIC  
SIGNATURE J. Murry NSB/5597

INTOXILYZER 8000  
Location: YCSO SELIGMAN  
Serial Number: 80-001734  
Core Version: 8105.48  
05/26/2013

10:21:14

---

Standard Lot#: 09312100A1  
Last Changed By: J. MURRAY #N56

---

QAS: J. MURRAY #N56  
YAVAPAI CSO

---

Operator: J. MURRAY #N56  
YAVAPAI CSO

---

Subject: TEST,SQAP  
DOB: 10/20/1960  
Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	10:22:08
Diagnostic Test	Pass	10:22:43
Air Blank	0.000	10:23:10
0.100 Cal Check	0.098	10:23:31
Air Blank	0.000	10:24:01
Subject Test	0.000	10:25:11
Air Blank	0.000	10:25:38
Wait	ABT*	
Air Blank	IPS**	10:26:15

\*Sequence Aborted

\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YCSO SELIGMAN  
Serial Number: 80-001734  
Core Version: 8105.48  
05/26/2013

10:26:29

---

Standard Lot#: 09312100A1  
Last Changed By: J. MURRAY #N56

---

QAS: J. MURRAY #N56  
YAVAPAI CSO

---

Operator: J. MURRAY #N56  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 10/20/1960  
Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	10:27:01
Air Blank	IPS**	10:27:08

\*RFI Detect  
\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YCSO SELIGMAN  
Serial Number: 80-001734  
Core Version: 8105.48  
05/26/2013

10:27:14

---

Standard Lot#: 09312100A1  
Last Changed By: J. MURRAY #N56

---

QAS: J. MURRAY #N56  
YAVAPAI CSO

---

Operator: J. MURRAY #N56  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 10/20/1960  
Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	10:28:10
Diagnostic Test	Pass	10:28:45
Air Blank	0.000	10:29:12
0.100 Cal Check	0.099	10:29:33
Air Blank	0.000	10:30:03
Subject Test	0.315*	10:33:10
Air Blank	0.000	10:33:45
Air Blank	0.000	10:34:12
Subject Test	XXX**	10:35:07
Air Blank	0.000	10:35:45

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Murray AGENCY Yavapai County Sheriff's Office  
DATE 8/26/2013 TIME 10:32:42  
INTOXILYZER SERIAL # 80-01734 LOCATION Seligman 103

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 100 AC.  
OR  
Pour a standard alcohol concentration solution        AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$ .
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.097 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray NSB/5597

DPS Form Exh G-2 (Rev 05-01)

SPECIALIST

8/26

INTOXILYZER SERIAL #

1. Ensure that

2. Intoxilyzer

3. Go to the

4. Air blank

5. Calibration

6. Air blank

7. Remove

8. Attach

9. Print

10. Sign

11. Date

12. Initials

13. Agency

14. Signature

15. Date

16. Initials

17. Agency

18. Signature

19. Date

20. Initials

21. Agency

INTOXILYZER 8000

Location: YCSO SELIGMAN

Serial Number: 80-001734

Core Version: 8105.48

05/26/2013

10:36:42

-----  
Standard Lot#: 09312100A1

Last Changed By: J. MURRAY #N56  
-----

QAS: J. MURRAY #N56

YAVAPAI CSO  
-----

Operator: J. MURRAY #N56

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	10:37:14
0.100 Cal Check	0.097	10:37:34
Air Blank	0.000	10:38:04

Preventative Maintenance  
Performed -- Timer Reset