

COPY

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Murray NSB AGENCY Yavapai County Sheriff's Office  
DATE 9/29/13 TIME 12:26:02  
INTOXILYZER SERIAL # 80-01734 LOCATION Seligman 103

- (+) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution        AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (+) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (+) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (+) 4. Air blank completed.
- (+) 5. Calibration check completed. Test results 0. 098 AC.
- (+) 6. Air blank completed.
- (+) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray NSB/5597

INTOXILYZER 8000

Location: YCSO SELIGMAN

Serial Number: 80-001734

Core Version: 8105.48

09/29/2013

12:26:02

-----  
Standard Lot#: 09312100A1

Last Changed By: J. MURRAY #N56  
-----

QAS: J. MURRAY #N56

YAVAPAI CSO  
-----

Operator: J. MURRAY #N56

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	12:26:34
0.100 Cal Check	0.098	12:26:55
Air Blank	0.000	12:27:24

Preventative Maintenance

Performed -- Timer Reset

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

TESTER NAME MURPHY AGENCY Yavapai County Sheriff's Office  
DATE 12/9/13 TIME 12:28:39  
INTOXILYZER SERIAL # 80-01734 LOCATION Seligman 103

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0. 000 AC.
- 2. Error Recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- 3. Pro sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 100 AC. Result 0. 096 AC.

Instrument operating properly and accurately. YES  NO

COMMENTS

09312100A1  
exp. - 6/1/2014  
800 P.S.I.  
 Display

DISTRICT  
SIGNATURE M. Murphy NSB/5597

DPS Form IGNA G-4 (Rev 05-01)

- Alcohol
- Breath
- Pro
- Standard
- Instrument
- Other

INTOXILYZER 8000  
Location: YCSO SELIGMAN  
Serial Number: 80-001734  
Core Version: 8105.48  
09/29/2013

12:28:39

---

Standard Lot#: 09312100A1  
Last Changed By: J. MURRAY #N56

---

QAS: J. MURRAY #N56  
YAVAPAI CSO

---

Operator: J. MURRAY 5597 #N56  
YAVAPAI YCSO

---

Subject: SQAP,TEST  
DOB: 10/20/1960  
Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	12:29:55
Diagnostic Test	Pass	12:30:30
Air Blank	0.000	12:30:58
0.100 Cal Check	0.097	12:31:18
Air Blank	0.000	12:31:48
Subject Test	0.000	12:32:18
Air Blank	0.000	12:32:45
Wait	ABT*	
Air Blank	RFI**	12:33:02

\*Sequence Aborted  
\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YCSO SELIGMAN  
Serial Number: 80-001734  
Core Version: 8105.48  
09/29/2013

12:33:11

---

Standard Lot#: 09312100A1  
Last Changed By: J. MURRAY #N56

---

QAS: J. MURRAY #N56  
YAVAPAI CSO

---

Operator: J. MURRAY 5597 #N56  
YAVAPAI YCSO

---

Subject: SQAP,TEST  
DOB: 10/20/1960  
Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	12:33:58
Air Blank	RFI**	12:34:05

\*Improper Sample

\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YCSO SELIGMAN  
Serial Number: 80-001734  
Core Version: 8105.48  
09/29/2013

12:34:34

---

Standard Lot#: 09312100A1  
Last Changed By: J. MURRAY #N56

---

QAS: J. MURRAY #N56  
YAVAPAI CSO

---

Operator: J. MURRAY 5597 #N56  
YAVAPAI YCSO

---

Subject: SQAP,TEST  
DOB: 10/20/1960  
Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	12:35:33
Diagnostic Test	Pass	12:36:09
Air Blank	0.000	12:36:36
0.100 Cal Check	0.097	12:36:57
Air Blank	0.000	12:37:26
Subject Test	0.375*	12:40:34
Air Blank	0.000	12:41:09
Air Blank	0.000	12:41:37
Subject Test	XXX**	12:42:11
Air Blank	0.000	12:42:48

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

NAME MURRAY AGENCY Yavapai County Sheriff's Office  
DATE 9/13 TIME 12:44:10  
INSTRUMENT SERIAL # 80-01734 LOCATION Seligman 103

- 1) Verify that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
a standard alcohol concentration solution                      AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.5^{\circ}\text{C}$ .
- 2) Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 3) Press the "D" button to go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made, press ENTER.
- 4) Calibration check completed. Test results 0.096 AC.
- 5) Calibration check completed.
- 6) Print the printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray NS6/5597

OPS Form E-2 (Rev 05-01)

- 1. Pour a small amount of simulator solution into the simulator.
- 2. Intoxilyzer
- 3. Go to the simulator
- 4. Intoxilyzer
- 5. Intoxilyzer
- 6. Intoxilyzer
- 7. Intoxilyzer

SIGNATURE J

INTOXILYZER 8000

Location: YCSO SELIGMAN

Serial Number: 80-001734

Core Version: 8105.48

09/29/2013

12:44:10

-----  
Standard Lot#: 09312100A1

Last Changed By: J. MURRAY #N56  
-----

QAS: J. MURRAY 5597 #N56

YAVAPAI YCSO  
-----

Operator: J. MURRAY 5597 #N56

YAVAPAI YCSO  
-----

Test	g/210L	Time
Air Blank	0.000	12:44:52
0.100 Cal Check	0.096	12:45:13
Air Blank	0.000	12:45:43

Preventative Maintenance  
Performed -- Timer Reset