

COPY

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Murphy NS6/5597 AGENCY Yavapai County Sheriff's Office  
DATE 7/8/2019 TIME ~~15:19:26~~ 15:19:26  
INTOXILYZER SERIAL # 80-01734 LOCATION Seligman

- (+) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution        AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (+) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (+) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (+) 4. Air blank completed.
- (+) 5. Calibration check completed. Test results 0.098 AC.
- (+) 6. Air blank completed.
- (+) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murphy NS6/5597

INTOXILYZER 8000

Location: YCSO SELIGMAN

Serial Number: 80-001734

Core Version: 8105.48

07/08/2014

15:18:25

-----  
Standard Lot#: 3391300A3

Last Changed By: J. MURRAY5597 #N56  
-----

QAS: J. MURRAY5597 #N56

YAVAPAI SHERIFFS  
-----

Operator: J. MURRAY5597 #N56

YAVAPAI SHERIFFS  
-----

Test	g/210L	Time
Air Blank	RFI*	15:18:52
Air Blank	0.000	15:19:20

\*RFI Detect

INTOXILYZER 8000  
Location: YCSO SELIGMAN  
Serial Number: 80-001734  
Core Version: 8105.48  
07/08/2014

15:19:26

-----  
Standard Lot#: 3391300A3  
Last Changed By: J. MURRAY5597 #N56  
-----

QAS: J. MURRAY5597 #N56  
YAVAPAI SHERIFFS  
-----

Operator: J. MURRAY5597 #N56  
YAVAPAI SHERIFFS  
-----

Test	g/210L	Time
Air Blank	0.000	15:19:58
0.100 Cal Check	0.098	15:20:18
Air Blank	0.000	15:20:48

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. Murray AGENCY Yavapai County Sheriff's Office  
DATE 7/8/2014 TIME 15:22:09  
INTOXILYZER SERIAL # 80-01734 LOCATION Seligman

(\*) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (\*) 1. Clock time check.  
(\*) 2. Date check.

OPERATIONAL TESTS

- (\*) 1. Alcohol-free subject test result 0. 000 AC.  
(\*) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
(\*) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(\*) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 097 AC.

Instrument is operating properly and accurately. YES X NO \_\_\_\_\_

COMMENTS

GAS SN# 33913100A3  
exp. 1/1/2016 1000 PST

SIGNATURE J. Murray NSB15597

INTOXILYZER 8000  
Location: YCSO SELIGMAN  
Serial Number: 80-001734  
Core Version: 8105.48  
07/08/2014

15:22:09

---

Standard Lot#: 3391300A3  
Last Changed By: J. MURRAY5597 #N56

---

QAS: J. MURRAY5597 #N56  
YAVAPAI SHERIFFS

---

Operator: J. MURRAY5597 #N56  
YAVAPAI SHERIFFS

---

Subject: SQAP,TEST  
DOB: 10/20/1960  
Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	15:23:06
Diagnostic Test	Pass	15:23:41
Air Blank	0.000	15:24:09
0.100 Cal Check	0.098	15:24:30
Air Blank	0.000	15:24:59
Subject Test	0.000	15:25:32
Air Blank	0.000	15:26:00
Wait	ABT*	
Air Blank	RFI**	15:26:14

\*Sequence Aborted

\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: YCSO SELIGMAN

Serial Number: 80-001734

Core Version: 8105.48

07/08/2014

15:26:20

---

Standard Lot#: 3391300A3

Last Changed By: J. MURRAY5597 #N56

---

QAS: J. MURRAY5597 #N56

YAVAPAI SHERIFFS

---

Operator: J. MURRAY5597 #N56

YAVAPAI SHERIFFS

---

Subject: SQAP, TEST

DOB: 10/20/1960

Sex: F

Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	15:27:00
Air Blank	RFI**	15:27:13

\*Improper Sample

\*\*RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: YCSO SELIGMAN  
Serial Number: 80-001734  
Core Version: 8105.48  
07/08/2014

15:27:39

---

Standard Lot#: 3391300A3  
Last Changed By: J. MURRAY5597 #N56

---

QAS: J. MURRAY5597 #N56  
YAVAPAI SHERIFFS

---

Operator: J. MURRAY5597 #N56  
YAVAPAI SHERIFFS

---

Subject: SQAP,TEST  
DOB: 10/20/1960  
Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	15:28:42
Diagnostic Test	Pass	15:29:17
Air Blank	0.000	15:29:45
0.100 Cal Check	0.097	15:30:06
Air Blank	0.000	15:30:35
Subject Test	0.239*	15:33:43
Air Blank	0.000	15:34:16
Air Blank	0.000	15:34:43
Subject Test	RGE**	15:35:16
Air Blank	0.000	15:35:52

\*Deficient Sample  
\*\*Range Exceeded

Not a Successfully Completed Test Sequence

INTOXILYZER 8000  
Location: YCSO SELIGMAN  
Serial Number: 80-001734  
Core Version: 8105.48  
07/08/2014 15:35:55

---

Standard Lot#: 3391300A3  
Last Changed By: J. MURRAY5597 #N56

---

OAS: J. MURRAY5597 #N56  
YAVAPAI SHERIFFS

---

Operator: J. MURRAY5597 #N56  
YAVAPAI SHERIFFS

---

Subject: SOAP,TEST  
DOB: 10/20/1960  
Sex: F Weight: 115

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	15:36:55
Diagnostic Test	Pass	15:37:30
Air Blank	0.000	15:37:57
0.100 Cal Check	0.096	15:38:18
Air Blank	0.000	15:38:47
Subject Test	XXX*	15:39:32
Air Blank	0.000	15:40:05

\*Invalid Sample  
Begin new deprivation period

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST M. MURPHY NSB/5597 AGENCY Yavapai County Sheriff's Office  
DATE 7/8/2014 TIME 15:40:53  
INTOXILYZER SERIAL # 80-01734 LOCATION Seligman

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution        AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 097 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murphy NSB/5597

INTOXILYZER 8000

Location: YCSO SELIGMAN

Serial Number: 80-001734

Core Version: 8105.48

07/08/2014

15:40:53

-----  
Standard Lot#: 3391300A3

Last Changed By: J. MURRAY5597 #N56  
-----

QAS: J. MURRAY5597 #N56

YAVAPAI SHERIFFS  
-----

Operator: J. MURRAY5597 #N56

YAVAPAI SHERIFFS  
-----

Test	g/210L	Time
Air Blank	0.000	15:41:30
0.100 Cal Check	0.097	15:41:51
Air Blank	0.000	15:42:20

Preventative Maintenance  
Performed -- Timer Reset