

COPY

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Murphy AGENCY Yavapai County Sheriff's Office
DATE 5/10/15 TIME 14:04:38
INTOXILYZER SERIAL # 80-01734 LOCATION Seligman

- (x) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution, 100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (x) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (x) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made, press ENTER.
- (x) 4. Air blank completed.
- (x) 5. Calibration check completed. Test results 0. 097 AC.
- (x) 6. Air blank completed.
- (x) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murphy NS675597

DPS Form Exh G-2 (Rev 05-01)

Tink #339/3100A3
11/2016
800PSI

INTOXILYZER 8000

Location: YCSO SELIGMAN

Serial Number: 80-001724

Core Version: S105.48

07/10/2015

14:04:38

Standard Lot#: 33913100A3

Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56

YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56

YAVAPAI SHERIFFS

| Test | g/210L | Time |
|-----------------|--------|----------|
| Air Blank | 0.000 | 14:05:42 |
| 0.100 Cal Check | 0.097 | 14:06:03 |
| Air Blank | 0.000 | 14:06:32 |

Preventative Maintenance
Performed -- Timer Reset