

DR 17-000607

COPY

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Scott Blakley Sr. AGENCY Yavapai County SO

DATE 01-08-17 TIME 23:21:01

INTOXILYZER SERIAL # 80-0734 LOCATION Seligman

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution N/A AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.098 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Scott Blakley Sr.

DPS Form Exh G-2 (Rev 05-01)

Tank Serial # 22315100A1

Exp 9-5-17

press 550 psi

INTOXILYZER 3000

Location: YCSO SELIGMAN

Serial Number: 80-001734

Core Version: 8105.48

01/05/2017

23:21:01

Standard Lot#: 22315100A1

Last Changed By: J. MURRAY5597 #N56

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	23:21:35
0.100 Cal Check	0.098	23:21:56
Air Blank	0.000	23:22:25

Preventative Maintenance
Performed -- Timer Reset

DR 17-000607

COPY

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Scott Bickley Sr. AGENCY Yavapai County SO
DATE 01-05-17 TIME 23:26:46
INTOXILYZER SERIAL # 80-01734 LOCATION Seligman

(X) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (X) 1. Clock time check.
- (X) 2. Date check.

OPERATIONAL TESTS

- (X) 1. Alcohol-free subject test result 0.000 AC.
- (X) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (X) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (X) 4. Standard Calibration Check standard 0.100 AC. Result 0.097 AC.

Instrument is operating properly and accurately. YES X NO _____

COMMENTS _____

SIGNATURE Scott Bickley Sr.

INTOXILYZER 8000

Location: YCSO SELIGMAN
Serial Number: 80-001734
Core Version: 8105.48
01/05/2017

23:26:46

Standard Lot#: 22315100A1
Last Changed By: J. MURRAY5597 #N56
OAS: S. BLAKLEY SR #L6861
YAV/APA1 CSO
Operator: S. BLAKLEY SR #L6861
YAV/APA1 CSO
Subject: SOAP, TEST
DOB: 10/20/1960
Sex: F Weight: 115
15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:28:13
Diagnostic Test	Pass	23:28:48
Air Blank	0.000	23:29:16
0.100 Cal Check	0.098	23:29:36
Air Blank	0.000	23:30:06
Subject Test	0.000	23:30:29
Air Blank	0.000	23:30:56
Wait	ABT*	
Air Blank	0.000	23:31:35

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YCSO SELIGMAN

Serial Number: 80-001734

Core Version: 8105.48

01/05/2017

23:32:32

Standard Lot#: 22315100A1

Last Changed By: J. MURRAY5597 #N56

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO

Subject: SQAP,TEST

DOB: 10/20/1960

Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	23:33:21
Air Blank	RFI**	23:33:40

*Improper Sample

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YCSO SELIGMAN

Serial Number: 80-001734

Core Version: 8105.48

01/05/2017

23:34:21

Standard Lot#: 22315100A1

Last Changed By: J. MURRAY5597

#N56

OAS: S. BLAKLEY SR

#L6861

YAVAPAI CSO

Operator: S. BLAKLEY SR

#L6861

YAVAPAI CSO

Subject: SOAP, TEST

DOB: 10/20/1960

Sex: F

Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:35:25
Diagnostic Test	Pass	23:36:00
Air Blank	0.000	23:36:28
0.100-Cal Check	0.098	23:36:49
Air Blank	0.000	23:37:18
Subject Test	0.015*	23:40:26
Air Blank	0.000	23:40:57
Air Blank	0.000	23:41:25
Subject Test	XXX**	23:41:48
Air Blank	0.000	23:42:21

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 0000

Location: YCSO SRELYGMAN

Serial Number: 80-001734

Core Version: 8105.48

01/05/2017

23:43:30

Standard Lot#: 22315100A1

Last Changed By: J. MURRAY5597 #N56

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	23:44:01
0.100 Cal Check	0.097	23:44:21
Air Blank	0.000	23:44:51

Preventative Maintenance
Performed -- Timer Reset