

DR 17-001167

COPY

EXHIBIT G-2
 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)
 ARIZONA DEPARTMENT OF PUBLIC SAFETY
 STANDARD QUALITY ASSURANCE PROCEDURES
 INTOXILYZER MODEL 8000
 STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST S. Blakely SA AGENCY Yavapai County SO
 DATE 3-30-17 TIME 23:29:17
 INTOXILYZER SERIAL # 80-01734 LOCATION Seligman YCSO

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
 OR
 Pour a standard alcohol concentration solution NA AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.098 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

DPS Form Exh G-2 (Rev 05-01) Tank Serial # 22315100A1
 Exp 9-5-2017
 Press 450 psi

RECEIVED
 APR 04 2017

DR 17-011167

IMMOKILYZER 8000
Location: YCSO SELIGMAN
Serial Number: 80-001734
Core Version: 8105.48
03/30/2017 23:29:17

Standard Lot#: 22315100A1
Last Changed By: J. MURRAY5597 #N56

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	23:29:53
0.100 Cal Check	0.098	23:30:14
Air Blank	0.000	23:30:44

Preventative Maistenance
Performed -- Timer Reset

DR 17-011167

Copy

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST S. Blakley AGENCY Yavapai County SO

DATE 3-30-17 TIME 23:32:42

INTOXILYZER SERIAL # 86-01734 LOCATION Seligman VESD

(X) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (X) 1. Clock time check.
- (X) 2. Date check.

OPERATIONAL TESTS

- (X) 1. Alcohol-free subject test result 0. 000 AC.
- (X) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (X) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (X) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 098 AC.

Instrument is operating properly and accurately. YES X NO

COMMENTS

SIGNATURE [Signature]

DPS Form Exh G-4 (Rev 05-01) Tank serial # 22315100A1

Exp 9-5-2017
press 450psi

DR 17-011167

Copy

INTOXILYZER 8000
Location: YCSO SELIGMAN
Serial Number: 80-001734
Cone Version: 8105.48
03/30/2017 23:32:42

Standard Lot#: 22315100A1
Last Changed By: J. MURRAY5597 #N56

OAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Subject: SOAP, TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:33:47
Diagnostic Test	Pass	23:34:22
Air Blank	0.000	23:34:49
0.100 Cal Check	0.098	23:35:10
Air Blank	0.000	23:35:40
Subject Test	0.000	23:35:59
Air Blank	0.000	23:36:26
Wait	ABT*	
Air Blank	0.000	23:37:05

*Sequence Aborted

Not a Successfully Completed Test Sequence

QR17-011167

INTOXILYZER 8000

Location: YCSO SELIGMAN

Serial Number: 80-001734

Core Version: 8105.48

03/30/2017

23:38:22

Standard Lot#: 22315100A1

Last Changed By: J. MURRAY5597 #N56

OAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Subject: SOAP, TEST

DOB: 10/20/1960

Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	23:39:19
Air Blank	RFI**	23:39:47

*Improper Sample

**RFI Detect

Not a Successfully Completed Test Sequence

DR 17-011167

INTOXILYZER 8000

Location: YCSO SELIGMAN

Serial Number: 80-001734

Core Version: 8105.48

03/30/2017

23:41:20

Standard Lot#: 22315100A1

Last Changed By: J. MURRAY5597 #N56

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Subject: SQAP, TEST

DOB: 10/20/1960

Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:42:19
Diagnostic Test	Pass	23:42:54
Air Blank	0.000	23:43:22
0.100 Cal Check	0.098	23:43:43
Air Blank	0.000	23:44:12
Subject Test	XXX*	23:46:02
Air Blank	0.000	23:46:38

*Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

DR 17-011167

INTOXILYZER 8000
Location: YCSO SELIGMAN
Serial Number: 80-001734
Core Version: 8105.48
03/30/2017

23:47:15

Standard Lot#: 22315100A1
Last Changed By: J. MURRAY5597 #N56

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Subject: SOAP, TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	23:48:15
Diagnostic Test	Pass	23:48:50
Air Blank	0.000	23:49:18
0.100 Cal Check	0.097	23:49:39
Air Blank	0.000	23:50:08
Subject Test	0.000*	23:53:16
Air Blank	0.000	23:53:47
Air Blank	0.000	23:54:15
Subject Test	DEF**	23:57:22
Air Blank	0.000	23:57:51

*Deficient Sample
**Deficient Sample

Not a Successfully Completed Test Sequence

DR 17-011167

INTOXILYZER 8000

Location: YCSO SELIGMAN

Serial Number: 80-001734

Core Version: 8105.48

03/30/2017

23:59:11

Standard Lot#: 22315100A1

Last Changed By: J. MURRAY5597 #N56

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	23:59:40
0.100 Cal Check	0.098	00:00:01
Air Blank	0.000	00:00:31

Preventative Maintenance
Performed -- Timer Reset

DR17-011167

COPY

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST S. Blakley Sr. AGENCY Yavapai County SO
DATE 3-31-17 TIME 00:01:33
INTOXILYZER SERIAL # 80-01734 LOCATION Seligman YESO

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution NA AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.095 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

DPS Form Exh G-2 (Rev 05-01)

Tank Serial # 22315100 A1

Exp 9-5-2017

press 450 psi

DIR 17-011167

INTOXLYZER 8000

Location: YCSO SELIGMAN

Serial Number: 80-001734

Core Version: 8105.48

03/31/2017

00:01:33

Standard Lot#: 22315100A1

Last Changed By: J. MURRAY5597 #N56

QAS: S. BLAKLEY SR #L6861

YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861

YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	00:02:04
0.100 Cal Check	0.095	00:02:25
Air Blank	0.000	00:02:54

Preventative Maintenance
Performed -- Timer Reset