

DR 17-014573

Copy

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Scott Blakley Sr. AGENCY Yavapai County SO
DATE 4-26-17 TIME 22:23:15
INTOXILYZER SERIAL # 80-01734 LOCATION Seligman

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
 Pour a standard alcohol concentration solution ^{OR} NA AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 098 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Scott Blakley Sr.

DPS Form Exh G-2 (Rev 05-01)

Tank Serial # 2231510041
Exp 9-5-2017
Pres. 400 psi

RECEIVED
MAY 04 2017
YAVAPAI COUNTY ATTORNEY

INTOXILYZER 8000
Location: YCSO SELIGMAN
Serial Number: 80-001734
Core Version: 8105.48
04/26/2017

22:23:15

Standard Lot#: 22315100A1
Last Changed By: J. MURRAY5597 #N56

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	22:24:13
0.100 Cal Check	0.098	22:24:34
Air Blank	0.000	22:25:03

Preventative Maintenance
Performed -- Timer Reset