

DR 18-009279

Copy

EXHIBIT G-2
 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)
 ARIZONA DEPARTMENT OF PUBLIC SAFETY
 STANDARD QUALITY ASSURANCE PROCEDURES
 INTOXILYZER MODEL 8000
 STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Scott Blakley Sr. AGENCY Yavapai County SO
 DATE 3-14-18 TIME 21:11:01
 INTOXILYZER SERIAL # 80-01734 LOCATION YCSO Seligman

- (x) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
 OR
 Pour a standard alcohol concentration solution NA AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (x) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (x) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (x) 4. Air blank completed.
- (x) 5. Calibration check completed. Test results 0. 099 AC.
- (x) 6. Air blank completed.
- (x) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE [Signature]

DPS Form Exh G-2 (Rev 05-01)

Tank Serial # 06717100A5
 Exp 5-5-2019
 Press 950 psi

DR 18-009279

INTOXILYZER 8000
Location: YCSO SELIGMAN
Serial Number: 80-001734
Core Version: 8105.48
03/14/2018 21:11:01

Standard Lot#: 0617100A5
Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	21:11:35
0.100 Cal Check	0.099	21:11:56
Air Blank	0.000	21:12:25

Preventative Maintenance
Performed -- Timer Reset