

DR18-012902

Copy

EXHIBIT G-2
 THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)
 ARIZONA DEPARTMENT OF PUBLIC SAFETY
 STANDARD QUALITY ASSURANCE PROCEDURES
 INTOXILYZER MODEL 8000
 STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Scott Blackley Sr. AGENCY Yavapai County SO
 DATE 4-10-18 TIME 21:09:35
 INTOXILYZER SERIAL # 80-01734 LOCATION YPSO Seligman

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
 Pour a standard alcohol concentration solution 0.100 AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.099 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Scott Blackley Sr.

DPS Form Exh G-2 (Rev 05-01)

Tank Serial # 0671700A5
 Exp 5-5-2019
 Pres. 950 psi

INTOXILYZER 8000
Location: YCSO SELIGMAN
Serial Number: 80-001734
Core Version: 8105.48
04/10/2018

21:09:35

Standard Lot#: 0617100A5
Last Changed By: S. BLAKLEY SR #L6861

QAS: S. BLAKLEY SR #L6861
YAVAPAI CSO

Operator: S. BLAKLEY SR #L6861
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	21:10:17
0.100 Cal Check	0.099	21:10:38
Air Blank	0.000	21:11:08

Preventative Maintenance
Performed -- Timer Reset

DR18-012902