

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Sgt. D. Jacobs AGENCY Camp Verde Marshals office
DATE 1/6/17 TIME _____
INTOXILYZER SERIAL # 80-001736 LOCATION CVMA Evidence room

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0.098 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____

Sgt C108

INTOXILYZER 8000
Location: CVMO
Serial Number: 80-001736
Core Version: 8105.48
01/06/2017 22:10:02

Standard Lot#: 09216100A1
Last Changed By: D. HUGHES #53

QAS: D. JACOBS #CV08
CAMP VERDE MO

Operator: D. JACOBS #CV08
CAMP VERDE MO

Test	g/210L	Time
Air Blank	0.000	22:10:42
0.100 Cal Check	0.098	22:11:03
Air Blank	0.000	22:11:32

Preventative Maintenance
Performed -- Timer Reset