

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co. S.O.  
DATE 8-30-08 TIME 1725  
INTOXILYZER SERIAL # 80-000289 LOCATION YCSO-MOBILE COMMAND

- OR) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.080 AC.  
OR) OR Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- OR) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- OR) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- OR) 4. Air blank completed.
- OR) 5. Calibration check completed. Test results 0. 0.073 AC.
- OR) 6. Air blank completed.
- OR) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Manuel Q. Raim #5-31 / OAS #33026

DPS Form Exh G-2 (Rev 05-01)

INTOXILYZER 8000  
Location: YCSO-MOBILE  
Serial Number: 80-000289  
08/30/2008 17:49:56

Standard Lot#: 6270021  
Last Changed By: D. RAISS #3693

OAS: D. RAISS #3693  
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	17:50:23
Cal Check	0.073	17:50:42
Air Blank	0.000	17:51:11

Manuel Q. Raim #5-31  
Operator's Signature  
OAS #33026

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co. S.O.  
DATE 8-31-08 TIME 1935  
INTOXILYZER SERIAL # 80-000289 LOCATION YCSO-MOBILE COMMAND (MAYER)

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0. 000 AC.
- 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 080 AC. Result 0. 075 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS All OK.

---

---

---

---

SIGNATURE Daniel J. Raim #5-31/OAS #33026

INTOXILYZER 8000  
Location: YCS0-MOBILE  
Serial Number: 80-000289  
08/31/2008 19:38:51

Standard Lot#: 6270021  
Last Changed By: D. RAISS #3693

OAS: D. RAISS #3693  
YAUAPPRI CS0

Operator: D. RAISS #3693  
YAUAPPRI CS0

Subject: SQPP,TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	19:40:18
Diagnostic Test	Pass	19:40:47
Air Blank	0.000	19:41:15
0.080 Cal Check	0.076	19:41:33
Air Blank	0.000	19:42:02
Subject Test	0.000	19:42:22
Air Blank	0.000	19:42:49
Five Minute Wait	RBT*	
Air Blank	0.000	19:43:24

\*Sequence Aborted

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: YCS0-MOBILE  
Serial Number: 80-000289  
08/31/2008 19:43:31

Standard Lot#: 6270021  
Last Changed By: D. RAISS #3693

OAS: D. RAISS #3693  
YAUAPPRI CS0

Operator: D. RAISS #3693  
YAUAPPRI CS0

Subject: SQPP,TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	19:44:00
Air Blank	0.000	19:44:29

\*Improper Sample

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: YCS0-MOBILE  
Serial Number: 80-000289  
08/31/2008 19:44:34

Standard Lot#: 6270021  
Last Changed By: D. RAISS #3693

OAS: D. RAISS #3693  
YAUAPPRI CS0

Operator: D. RAISS #3693  
YAUAPPRI CS0

Subject: SQPP,TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	19:45:10
Air Blank	0.000	19:45:38

\*RFI Detect

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: YCS0-MOBILE  
Serial Number: 80-000289  
08/31/2008 19:45:45

Standard Lot#: 6270021  
Last Changed By: D. RAISS #3693

OAS: D. RAISS #3693  
YAUAPPRI CS0

Operator: D. RAISS #3693  
YAUAPPRI CS0

Subject: SQPP,TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	19:46:27
Diagnostic Test	Pass	19:46:56
Air Blank	0.000	19:47:23
0.080 Cal Check	0.077	19:47:42
Air Blank	0.000	19:48:11
Subject Test	0.124*	19:51:19
Air Blank	0.000	19:51:51
Air Blank	0.000	19:52:18
Subject Test	XXX**	19:52:42
Air Blank	0.000	19:53:15

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

Not a Successfully  
Completed  
Test Sequence

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI CO. S.O.  
DATE 8-31-08 TIME 1935

INTOXILYZER SERIAL # 80-000289 LOCATION YCSO-MOBILE COMMAND (MAYER)

- 06 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 080AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- 06 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- 06 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- 06 4. Air blank completed.
- 06 5. Calibration check completed. Test results 0. 075 AC.
- 06 6. Air blank completed.
- 06 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Dan Raiss #5-31 / OAS #33026

DPS Form Exh G-2 (Rev 05-01)

INTOXILYZER 8000  
Location: YCSO-MOBILE  
Serial Number: 80-000289  
08/31/2008 19:53:31

Standard Lot#: 6270021  
Last Changed By: D. RAISS #3693

OAS: D. RAISS #3693  
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	19:53:57
Cal Check	0.075	19:54:16
Air Blank	0.000	19:54:45

Dan Raiss #5-31  
Operator's Signature  
OAS #33026

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co. S.O.  
DATE 8-30-08 TIME 1725  
INTOXILYZER SERIAL # 80-000289 LOCATION YCSO - MOBILE COMMAND

- (OK) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.080 AC.  
OR  
Pour a standard alcohol concentration solution X AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (OK) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (OK) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (OK) 4. Air blank completed.
- (OK) 5. Calibration check completed. Test results 0. 0.074 AC.
- (OK) 6. Air blank completed.
- (OK) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Daniel A. Raim #5-31/OAS #33026

DPS Form Exh G-2 (Rev 05-01)

CAL CHECK - NEW DRY GAS BOTTLE  
EXPIRES 9-29-08  
LOT No. 627002I  
0.080% STANDARD

INTOXILYZER 8000  
Location: YCSO-MOBILE  
Serial Number: 80-000289  
08/30/2008 17:26:25

Standard Lot#: 612011  
Last Changed By: J. TRAINING #99999

OAS: J. TRAINING #99999  
AZ OPS

Operator: D. RAISS #3693  
YAUAPPAL CSO

Subject: SOPP TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:27:07
Diagnostic Test	Pass	17:27:36
Air Blank	0.000	17:28:03
0.100 Cal Check	0.074*	17:28:22
Air Blank	0.000	17:28:56

\*Calibration Check Out of Tolerance

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: YCSO-MOBILE  
Serial Number: 80-000289  
08/30/2008 17:30:07

Standard Lot#: 6270021  
Last Changed By: D. RAISS #3693

OAS: D. RAISS #3693  
YAUAPPAL CSO

Test	g/210L	Time
Air Blank	0.000	17:30:35
Cal Check	0.074	17:30:53
Air Blank	0.000	17:31:23

INTOXILYZER 8000  
Location: YCSO-MOBILE  
Serial Number: 80-000289  
08/30/2008 17:33:18

Standard Lot#: 6270021  
Last Changed By: D. RAISS #3693

OAS: D. RAISS #3693  
YAUAPPAL CSO

Test	g/210L	Time
Air Blank	0.000	17:33:45
Cal Check	0.000	17:34:04
Air Blank	0.000	17:34:31

INTOXILYZER 8000  
Location: YCSO-MOBILE  
Serial Number: 80-000289  
08/30/2008 17:34:40

Standard Lot#: 6270021  
Last Changed By: D. RAISS #3693

OAS: D. RAISS #3693  
YAUAPPAL CSO

Test	g/210L	Time
Air Blank	0.000	17:35:07
Cal Check	0.074	17:35:26
Air Blank	0.000	17:35:55

BOTTLE OFF

*David J. Raimo #5-31*  
Operator's Signature  
OAS #33026

*David J. Raimo #5-31*  
Operator's Signature  
OAS #33026

*David J. Raimo #5-31*  
Operator's Signature  
OAS #33026

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST DAN RAISS AGENCY YAVAPAI Co. S.O.  
DATE 8-30-08 TIME 1725  
INTOXILYZER SERIAL # 80-000289 LOCATION YCSO-Command Post  
CHINO VALLEY

(P) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (P) 1. Clock time check.  
(P) 2. Date check.

OPERATIONAL TESTS

- (P) 1. Alcohol-free subject test result 0. 000 AC.  
(P) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed  
(P) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.  
(P) 4. Standard Calibration Check standard 0. 080 AC. Result 0. 073 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

NEW DRY GAS BOTTLE. UPDATED INFO BUT NOT STANDARD OF .080. CAL CHECK  
OF .074 IS NOT OUT OF TOLERANCE. 2<sup>ND</sup> CAL CHECK OF .074 IS ALSO GOOD. CAL CHECK  
OF .000% IS GOOD- DRY GAS WAS REMOVED. CAL CHECK .074% BOTTLE BACK ATTACHED.  
SQAP - ALL OK. JUST NOTE STANDARD IS .080%.

SIGNATURE Daniel J. Raiss #5-31/QAS #33026

INTOXILYZER 8000  
Location: YCSO-MOBILE  
Serial Number: 80-000289  
08/30/2008 17:36:11

Standard Lot#: 6270021  
Last Changed By: D. RAISS #3693

OAS: D. RAISS #3693  
YAUAPP1 CSO

Operator: D. RAISS #3693  
YAUAPP1 CSO

Subject: SOAP, TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:37:03
Diagnostic Test	Pass	17:37:31
Air Blank	0.000	17:37:59
0.080 Cal Check	0.072	17:38:17
Air Blank	0.000	17:38:47
Subject Test	0.000	17:39:06
Air Blank	0.000	17:39:33
Five Minute Wait	ABT*	
Air Blank	0.000	17:40:08

\*Sequence Aborted

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: YCSO-MOBILE  
Serial Number: 80-000289  
08/30/2008 17:41:15

Standard Lot#: 6270021  
Last Changed By: D. RAISS #3693

OAS: D. RAISS #3693  
YAUAPP1 CSO

Operator: D. RAISS #3693  
YAUAPP1 CSO

Subject: SOAP, TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RF1*	17:41:40
Air Blank	0.000	17:42:08

\*RF1 Detect

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: YCSO-MOBILE  
Serial Number: 80-000289  
08/30/2008 17:42:15

Standard Lot#: 6270021  
Last Changed By: D. RAISS #3693

OAS: D. RAISS #3693  
YAUAPP1 CSO

Operator: D. RAISS #3693  
YAUAPP1 CSO

Subject: SOAP, TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	17:42:57
Diagnostic Test	Pass	17:43:26
Air Blank	0.000	17:43:53
0.080 Cal Check	0.074	17:44:12
Air Blank	0.000	17:44:41
Subject Test	0.122*	17:47:49
Air Blank	0.000	17:48:21
Air Blank	0.000	17:48:48
Subject Test	XXX**	17:49:04
Air Blank	0.000	17:49:37

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

Not a Successfully  
Completed  
Test Sequence

INTOXILYZER 8000  
Location: YCSO-MOBILE  
Serial Number: 80-000289  
08/30/2008 17:40:13

Standard Lot#: 6270021  
Last Changed By: D. RAISS #3693

OAS: D. RAISS #3693  
YAUAPP1 CSO

Operator: D. RAISS #3693  
YAUAPP1 CSO

Subject: SOAP, TEST,  
DOB: 01/01/1901  
Sex: M Weight: 1

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	17:40:41
Air Blank	0.000	17:41:09

\*Improper Sample

Not a Successfully  
Completed  
Test Sequence