

COPY

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. MURRAY NSG AGENCY Yavapai County Sheriff's
DATE 10/21/11 TIME 9:34:07
INTOXILYZER SERIAL # 80-000289 LOCATION Prescott

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (+) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
(x) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
(+) 4. Air blank completed.
(+) 5. Calibration check completed. Test results 0.097 AC.
(+) 6. Air blank completed.
(x) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. MURRAY NSG

INTOXILYZER 8000

Location: YAVAPAI CO. SHERIFF

Serial Number: 80-000289

Core Version: 8105.48

10/23/2011

09:34:07

Standard Lot#: 01011100A1

Last Changed By: J. MURRAY #N56

QAS: J. MURRAY #N56

YAVAPAI YCSO

Operator: J. MURRAY #N56

YAVAPAI YCSO

Test g/210L Time

Air Blank 0.000 09:34:42
0.100 Cal Check 0.097 09:35:02
Air Blank 0.000 09:35:32

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. Murray NSB AGENCY Yavapai County Sheriff's
DATE 10/23/11 TIME 09:35:46
INTOXILYZER SERIAL # 80-000289 LOCATION Prescott

(X) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (X) 1. Clock time check.
- (X) 2. Date check.

OPERATIONAL TESTS

- (X) 1. Alcohol-free subject test result 0. 000 AC.
- (X) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (X) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (X) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 095 AC.

Instrument is operating properly and accurately. YES X NO

COMMENTS

950 PSI
Tank # 81011100A1
exp. 2/1/2013

SIGNATURE J. Murray NSB

INTOXILYZER 8000

Location: YAVAPAI CO. SHERIFF

Serial Number: 80-000289

Core Version: 8105.48

10/23/2011

09:35:46

Standard Lot#: 01011100A1

Last Changed By: J. MURRAY

#N56

QAS: J. MURRAY

#N56

YAVAPAI YCSO

Operator: J. MURRAY

#N56

YAVAPAI YCSO

Subject: SQAP, TEST

DOB: 10/20/1960

Sex: F

Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:36:47
Diagnostic Test	Pass	09:37:22
Air Blank	0.000	09:37:49
0.100 Cal Check	0.095	09:38:10
Air Blank	0.000	09:38:39
Subject Test	0.000	09:38:56
Air Blank	0.000	09:39:23
Wait	ABT*	
Air Blank	RFI**	09:39:37

*Sequence Aborted

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI CO. SHERIFF

Serial Number: 80-000289

Core Version: 8105.48

10/23/2011

09:39:44

Standard Lot#: 01011100A1

Last Changed By: J. MURRAY

#N56

QAS: J. MURRAY

#N56

YAVAPAI YCSO

Operator: J. MURRAY

#N56

YAVAPAI YCSO

Subject: SQAP, TEST

DOB: 10/20/1960

Sex: F

Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	IPS*	09:40:28
Air Blank	RFI**	09:40:36

*Improper Sample

**RFI Detect

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI CO. SHERIFF

Serial Number: 80-000289

Core Version: 8105.48

10/23/2011

09:40:40

Standard Lot#: 01011100A1

Last Changed By: J. MURRAY

#N56

QAS: J. MURRAY

#N56

YAVAPAI YCSO

Operator: J. MURRAY

#5597N5

YAVAPAI YCSO

Subject: SQAP, TEST

DOB: 10/20/1960

Sex: F

Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:42:09
Diagnostic Test	Pass	09:42:44
Air Blank	0.000	09:43:11
0.100 Cal Check	0.096	09:43:32
Air Blank	0.000	09:44:01
Subject Test	0.136*	09:47:09
Air Blank	0.000	09:47:40
Air Blank	0.000	09:48:08
Subject Test	XXX**	09:48:37
Air Blank	0.000	09:49:10

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. MURRAY NS6 AGENCY Yavapai County Sheriff's
DATE 10/23/11 TIME 09:49:48
INTOXILYZER SERIAL # 80-000289 LOCATION Prescott

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 160 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 095 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. MURRAY NS6

INTOXILYZER 8000

Location: YAVAPAI CO. SHERIFF

Serial Number: 80-000289

Core Version: 8105.48

10/23/2011

09:49:48

Standard Lot#: 01011100A1

Last Changed By: J. MURRAY #N56

QAS: J. MURRAY #5597N56YYAVAPAI YCSO

YAVAPAI YCSO

Operator: J. MURRAY #5597N56YYAVAPAI YCSO

YAVAPAI YCSO

Test	g/210L	Time
Air Blank	0.000	09:50:27
0.100 Cal Check	0.095	09:50:48
Air Blank	0.000	09:51:18

Preventative Maintenance
Performed -- Timer Reset