

COPY

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. Murray NSB/5597 AGENCY Yavapai County Sheriff's Office
DATE 12/18/11 TIME 13:25:58
INTOXILYZER SERIAL # 80-000289 LOCATION Prescott 103

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0.095 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

J. Murray NSB

INTOXILYZER 8000

Location: YAVAPAI CO. SHERIFF

Serial Number: 80-000289

Core Version: 8105.48

12/18/2011

13:25:58

Standard Lot#: 01011100A1

Last Changed By: J. MURRAY #N56

QAS: J. MURRAY #N56

YAVAPAI YCSO

Operator: J. MURRAY #N56

YAVAPAI YCSO

Test	g/210L	Time
Air Blank	0.000	13:26:32
0.100 Cal Check	0.095	13:26:53
Air Blank	0.000	13:27:22

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST J. M. MURPHY NSB/5597 AGENCY Yavapai County Sheriff's Office
DATE 12/18/11 TIME 13:28:00
INTOXILYZER SERIAL # 80-000289 LOCATION Prescott 103

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

1. Clock time check.
 2. Date check.

OPERATIONAL TESTS

1. Alcohol-free subject test result 0. 000 AC.
 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
 4. Standard Calibration Check standard 0. 100 AC. Result 0. 095 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

0101100A1 exp 2/1/13 100
850 PST +/-

SIGNATURE J. M. MURPHY NSB

INTOXILYZER 8000

Location: YAVAPAI CO. SHERIFF

Serial Number: 80-000289

Core Version: 8105.48

12/18/2011

13:28:00

Standard Lot#: 01011100A1

Last Changed By: J. MURRAY

#N56

QAS: J. MURRAY

#N56

YAVAPAI YCSO

Operator: J. MURRAY

#N56

YAVAPAI YCSO

Subject: SQAP,TEST

DOB: 10/20/1960

Sex: F

Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:28:53
Diagnostic Test	Pass	13:29:28
Air Blank	0.000	13:29:56
0.100 Cal Check	0.096	13:30:17
Air Blank	0.000	13:30:46
Subject Test	0.000	13:31:07
Air Blank	0.000	13:31:34
Wait	ABT*	
Air Blank	IPS**	13:31:58

*Sequence Aborted

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI CO. SHERIFF

Serial Number: 80-000289

Core Version: 8105.48

12/18/2011

13:32:10

Standard Lot#: 01011100A1

Last Changed By: J. MURRAY

#N56

QAS: J. MURRAY

#N56

YAVAPAI YCSO

Operator: J. MURRAY

#N56

YAVAPAI YCSO

Subject: SQAP, TEST

DOB: 10/20/1960

Sex: F

Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	13:33:00
Air Blank	IPS**	13:33:09

*RFI Detect

**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: YAVAPAI CO. SHERIFF

Serial Number: 80-000289

Core Version: 8105.48

12/18/2011

13:33:13

Standard Lot#: 01011100A1

Last Changed By: J. MURRAY

#N56

QAS: J. MURRAY

#N56

YAVAPAI YCSO

Operator: J. MURRAY

#N56

YAVAPAI YCSO

Subject: SQAP,TEST

DOB: 10/20/1960

Sex: F

Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:34:08
Diagnostic Test	Pass	13:34:43
Air Blank	0.000	13:35:11
0.100 Cal Check	0.096	13:35:32
Air Blank	0.000	13:36:01
Subject Test	0.285*	13:39:08
Air Blank	0.000	13:39:40
Air Blank	0.000	13:40:07
Subject Test	XXX**	13:40:39
Air Blank	0.000	13:41:11

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST J. MURPHY NSB/5597 AGENCY Yavapai County Sheriff's Office
DATE 12/18/11 TIME 13:41:27
INTOXILYZER SERIAL # 80-000289 LOCATION Pinco #103

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (0) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 095 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. MURPHY NSB

INTOXILYZER 8000
Location: YAVAPAI CO. SHERIFF
Serial Number: 80-000289
Core Version: 8105.48
12/18/2011 13:41:27

Standard Lot#: 01011100A1
Last Changed By: J. MURRAY #N56

QAS: J. MURRAY #N56
YAVAPAI YCSO

Operator: J. MURRAY #N56
YAVAPAI YCSO

Test	g/210L	Time
Air Blank	0.000	13:42:03
0.100 Cal Check	0.095	13:42:24
Air Blank	0.000	13:42:53

Preventative Maintenance
Performed -- Timer Reset