

**COPY**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOODY AGENCY YAVAPAI CO. S.O.  
DATE 05/22/2012 TIME 0914  
INTOXILYZER SERIAL # 80-000289 LOCATION YCSO DUT TRAILER

1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
4. Air blank completed.
5. Calibration check completed. Test results 0. 093 AC.
6. Air blank completed.
7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Logan Moody 67486

**COPY**

INTOXILYZER 8000  
Location: YCSO DUI TRAILER  
Serial Number: 80-000289  
Core Version: 8105.48  
05/22/2012

09:14:43

-----  
Standard Lot#: 01011100A1  
Last Changed By: J. MURRAY #N56  
-----

QAS: L. MOODY #7486  
YAVAPAI CSO  
-----

Operator: L. MOODY #7486  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	09:15:31
0.100 Cal Check	0.093	09:15:52
Air Blank	0.000	09:16:21

Preventative Maintenance  
Performed -- Timer Reset

**COPY**

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST LOGAN MOODY AGENCY YAVAPAI CO. S.O.

DATE 05/22/2012 TIME 0919

INTOXILYZER SERIAL # 80-000289 LOCATION YCSO DUI TRAILER

(  ) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (  ) 1. Clock time check.
- (  ) 2. Date check.

OPERATIONAL TESTS

- (  ) 1. Alcohol-free subject test result 0. 000 AC.
- (  ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (  ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (  ) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 094 AC.

Instrument is operating properly and accurately. YES  NO

COMMENTS

---



---



---



---

SIGNATURE Logan Moody 6748

**COPY**

INTOXILYZER 8000  
Location: YCSO DUI TRAILER  
Serial Number: 80-000289  
Core Version: 8105.48  
05/22/2012 09:19:18

---

Standard Lot#: 01011100A1  
Last Changed By: J. MURRAY #N56

---

QAS: L. MOODY #7486  
YAVAPAI CSO

---

Operator: L. MOODY #7486  
YAVAPAI CSO

---

Subject: SQAP, TEST  
DOB: 01/01/2000  
Sex: M Weight: 190

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	09:20:14
Diagnostic Test	Pass	09:20:49
Air Blank	0.000	09:21:17
0.100 Cal Check	0.093	09:21:38
Air Blank	0.000	09:22:07
Subject Test	0.000	09:22:32
Air Blank	ABT*	09:22:41
Air Blank	RFI**	09:22:48

\*Sequence Aborted

\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

**COPY**

INTOXILYZER 8000  
Location: YCSO DUI TRAILER  
Serial Number: 80-000289  
Core Version: 8105.48  
05/22/2012 09:23:08

---

Standard Lot#: 01011100A1  
Last Changed By: J. MURRAY #N56

---

QAS: L. MOODY #7486  
YAVAPAI CSO

---

Operator: L. MOODY #7486  
YAVAPAI CSO

---

Subject: SQAP, TEST  
DOB: 01/01/2000  
Sex: M Weight: 190

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	09:24:02
Diagnostic Test	Pass	09:24:37
Air Blank	0.000	09:25:05
0.100 Cal Check	0.094	09:25:25
Air Blank	0.000	09:25:55
Subject Test	0.077*	09:29:02
Air Blank	0.000	09:29:34
Air Blank	0.000	09:30:01
Subject Test	XXX**	09:30:34
Air Blank	IPS***	09:30:42

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

\*\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

**COPY**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOODY AGENCY YAVAPAI CO. S.O.  
DATE 05 / 22 / 2012 TIME 0931  
INTOXILYZER SERIAL # 80-000289 LOCATION YCSO DUI TRAILER

- () 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- () 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- () 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check completed. Test results 0. 094 AC.
- () 6. Air blank completed.
- () 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE LMoody 27188

**COPY**

INTOXILYZER 8000  
Location: YCSO DUI TRAILER  
Serial Number: 80-000289  
Core Version: 8105.48  
05/22/2012 09:31:36

-----  
Standard Lot#: 01011100A1  
Last Changed By: J. MURRAY #N56  
-----

QAS: L. MOODY #7486  
YAVAPAI CSO  
-----

Operator: L. MOODY #7486  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	AMB*	09:32:06
Air Blank	0.000	09:32:34

\*Ambient Fail

**COPY**

INTOXILYZER 8000  
Location: YCSO DUI TRAILER  
Serial Number: 80-000289  
Core Version: 8105.48  
05/22/2012 09:32:50

-----  
Standard Lot#: 01011100A1  
Last Changed By: J. MURRAY #N56  
-----

QAS: L. MOODY #7486  
YAVAPAI CSO  
-----

Operator: L. MOODY #7486  
YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	09:33:20
0.100 Cal Check	0.094	09:33:40
Air Blank	0.000	09:34:10

Preventative Maintenance  
Performed -- Timer Reset