

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOODY 7486 AGENCY YAVAPAI CO. S.O.

DATE 3-2-2014 TIME 1453

INTOXILYZER SERIAL # 80-000289 LOCATION YCSO - MAYER

- ~~(M)~~ 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
~~Pour a standard alcohol concentration solution AC into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- (M) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (M) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (M) 4. Air blank completed.
- (M) 5. Calibration check completed. Test results 0. 094 AC.
- (M) 6. Air blank completed.
- (M) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Moody 7486

INTOXILYZER 8000

Location: YCSO DUI TRAILER

Serial Number: 80-000289

Core Version: 8105.48

03/02/2014

14:53:48

-----  
Standard Lot#: 13712100A2

Last Changed By: L. MOODY #7486  
-----

QAS: L. MOODY #7486

YAVAPAI CSO  
-----

Operator: L. MOODY #7486

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	14:54:17
0.100 Cal Check	0.094	14:54:38
Air Blank	0.000	14:55:07

Preventative Maintenance  
Performed -- Timer Reset

EXHIBIT G-4  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST LOGAN MOODY 7486 AGENCY YAVAPAI CO. S.O.

DATE 3-2-2014 TIME 1456

INTOXILYZER SERIAL # 80-000289 LOCATION YCSO. MACER

(67) 1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (67) 1. Clock time check.
- (67) 2. Date check.

OPERATIONAL TESTS

- (67) 1. Alcohol-free subject test result 0. 000 AC.
- (67) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (67) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (67) 4. Standard Calibration Check standard 0. 100 AC. Result 0. 093 AC.

Instrument is operating properly and accurately. YES (67) NO \_\_\_\_\_

COMMENTS

600 psi

SIGNATURE Moody 7486

INTOXILYZER 8000  
Location: YCSO DUI TRAILER  
Serial Number: 80-000289  
Core Version: 8105.48  
03/02/2014

14:56:13

---

Standard Lot#: 13712100A2  
Last Changed By: L. MOODY #7486

---

QAS: L. MOODY #7486  
YAVAPAI CSO

---

Operator: L. MOODY #7486  
YAVAPAI CSO

---

Subject: SQAP,TEST  
DOB: 01/01/2000  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	14:57:21
Diagnostic Test	Pass	14:57:56
Air Blank	0.000	14:58:24
0.100 Cal Check	0.094	14:58:45
Air Blank	0.000	14:59:14
Subject Test	0.000	14:59:47
Air Blank	ABT*	14:59:54
Air Blank	RFI**	15:00:02

\*Sequence Aborted

\*\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: YCSO DUI TRAILER  
Serial Number: 80-000289  
Core Version: 8105.48  
03/02/2014

15:00:06

---

Standard Lot#: 13712100A2  
Last Changed By: L. MOODY #7486

---

QAS: L. MOODY #7486  
YAVAPAI CSO

---

Operator: L. MOODY #7486  
YAVAPAI CSO

---

Subject: SQAP,TEST'  
DOB: 01/01/2000  
Sex: M Weight: 200

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	15:00:54
Diagnostic Test	Pass	15:01:29
Air Blank	0.000	15:01:57
0.100 Cal Check	0.095	15:02:18
Air Blank	0.000	15:02:47
Subject Test	0.089*	15:05:54
Air Blank	0.000	15:06:26
Air Blank	0.000	15:06:53
Subject Test	XXX**	15:07:15
Air Blank	IPS***	15:07:24

\*Deficient Sample  
\*\*Invalid Sample  
    Begin new deprivation period  
\*\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

EXHIBIT G-2  
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST LOGAN MOODY 7486 AGENCY YAVAPAI CO. S.O.

DATE 2-3-2014 TIME 1507

INTOXILYZER SERIAL # 80-000289 LOCATION YCSO - MAGER

- (67) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.  
OR  
~~Pour a standard alcohol concentration solution \_\_\_\_\_ AC into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach  $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$~~
- (67) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (67) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (67) 4. Air blank completed.
- (67) 5. Calibration check completed. Test results 0.093 AC.
- (67) 6. Air blank completed.
- (67) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Logan Moody 7486

INTOXILYZER 8000

Location: YCSO DUI TRAILER

Serial Number: 80-000289

Core Version: 8105.48

03/02/2014

15:07:38

-----  
Standard Lot#: 13712100A2

Last Changed By: L. MOODY #7486  
-----

QAS: L. MOODY #7486

YAVAPAI CSO  
-----

Operator: L. MOODY #7486

YAVAPAI CSO  
-----

Test	g/210L	Time
Air Blank	0.000	15:08:08
0.100 Cal Check	0.093	15:08:28
Air Blank	0.000	15:08:58

Preventative Maintenance  
Performed -- Timer Reset