

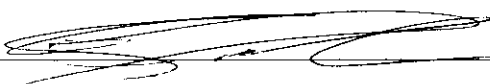
EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST RT POOLE AGENCY VALUAPPE COUNTY S.O
DATE 4/23/2014 TIME 11:04
INTOXILYZER SERIAL # 80-00289 LOCATION CAMP VENCE JAIL

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
~~Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C~~
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 096 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE  POOLE RTS OAS # 33629

INTOXILYZER 8000
Location: YCSO SPARE
Serial Number: 80-000289
Core Version: 8105.48
04/24/2014 11:03:36

Standard Lot#: 13712100A2
Last Changed By: L. MOODY #7486

QAS: R. POOLE #9014
YAVAPAI CSO

Operator: R. POOLE #9014
YAVAPAI CSO

Test	g/210L	Time
Air Blank	0.000	11:04:07
0.100 Cal Check	0.096	11:04:28
Air Blank	0.000	11:04:58

Preventative Maintenance
Performed -- Timer Reset