

#1

FILE COPY

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Murray AGENCY Yavapai County Sheriff's Office
DATE 9/1/14 TIME 09:35:07
INTOXILYZER SERIAL # 80-060289 LOCATION Spore

- (X) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (X) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (X) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (X) 4. Air blank completed.
- (X) 5. Calibration check completed. Test results 0. 094 AC.
- (X) 6. Air blank completed.
- (X) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murray NS6 15597

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST _____

FA

AGENCY _____

DATE _____

TIME _____

INTOXILYZER S _____

LOCATION _____

() 1. Ensure

INTOXILYZER 8000
Location: YCSO SPARE
Serial Number: 80-000289
Core version: 8105.48
09/01/2014 09:35:07

contains a standard alcohol concentration solution _____ AC.
OR

Pour a
simulat
0.2°C

_____ AC, into a clean dry simulator and assemble the
. Turn on the simulator and allow temperature to reach 34°C ±

() 2. Intoxily

Standard Lot#: 33913100A3
Last Changed By: J. MURRAY5597 #N56

TO START”
r dry control test or “W” for wet control test. After selection is

() 3. Go to th

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

_____ AC.

made p

() 4. Air bla

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Completed checklist.

() 5. Calibra

() 6. Air bla

() 7. Remov

Test	g/210L	Time
Air Blank	0.000	09:35:40
0.100 Cal Check	0.094	09:36:00
Air Blank	0.000	09:36:30

SIGNATURE _____

DPS Form Exh G-

Preventative Maintenance
Performed -- Timer Reset

EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Murphy AGENCY Yavapai County Sheriff's Office
DATE 9/1/2014 TIME 09:38:73
INTOXILYZER SERIAL # 80-000289 LOCATION Spare

1. Display Reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- 1. Clock time check.
- 2. Date check.

OPERATIONAL TESTS

- 1. Alcohol-free subject test result 0. 000 AC.
- 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- 4. Standard Calibration Check standard 0. 100 AC. Result 0. 094 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

Replaced Gas Bottle 33913100A3 11/1/2016 71000 PSI

SIGNATURE Murphy NSB/5597

**EXHIBIT G-4
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)**

ARIZONA DEPARTMENT OF PUBLIC SAFETY

**STANDARD QUALITY ASSURANCE PROGRAM
INTOXILYZER MODEL 8000**

INTOXILYZER 8000
Location: YCSO SPARE
Serial Number: 80-000289
Core Version: 8105.48
09/01/2014 09:38:13

Standard Lot#: 33913100A3
Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Subject: SOAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:39:09
Diagnostic Test	Pass	09:39:46
Air Blank	0.000	09:40:14
0.100 Cal Check	0.096	09:40:34
Air Blank	0.000	09:41:04
Subject Test	0.000	09:41:33
Air Blank	0.000	09:42:00
Wait	RBT*	
Air Blank	RFI**	09:42:16

*Sequence Aborted
**RFI Detect

Not a Successfully
Completed
Test Sequence

STAIR

INTOXILYZER 8000
Location: YCSO SPARE
Serial Number: 80-000289
Core Version: 8105.48
09/01/2014 09:42:55

I BUTTC

Standard Lot#: 33913100A3
Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Test result
of system
is fully Co
mpletion syst
is fully Co
mpletion print
Check sta

ly and acc

Test	g/210L	Time
Air Blank	IPS*	09:43:07
Air Blank	RFI**	09:43:45

*Improper Sample
**RFI Detect

Not a Successfully
Completed
Test Sequence

11)

INTOXILYZER 8000
Location: YCSO SPARE
Serial Number: 80-000289
Core Version: 8105.48
09/01/2014 09:44:05

Standard Lot#: 33913100A3
Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Subject: SOAP,TEST
DOB: 10/20/1960
Sex: F Weight: 115

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:45:04
Diagnostic Test	Pass	09:45:41
Air Blank	0.000	09:46:08
0.100 Cal Check	0.095	09:46:29
Air Blank	0.000	09:46:59
Subject Test	0.273*	09:50:06
Air Blank	0.000	09:50:38
Air Blank	0.000	09:51:05
Subject Test	XXX**	09:51:50
Air Blank	0.000	09:52:24

*Deficient Sample
**Invalid Sample
Begin new deprivation period

Not a Successfully
Completed
Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST MURPHY AGENCY Yavapai County Sheriff's Office
DATE 9/11/14 TIME 09:52:49
INTOXILYZER SERIAL # 80-000289 LOCATION Spire

- (x) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution .100 AC.
OR
Pour a standard alcohol concentration solution AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (x) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (x) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (x) 4. Air blank completed.
- (x) 5. Calibration check completed. Test results 0. 094 AC.
- (x) 6. Air blank completed.
- (x) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE J. Murphy NSB/5597

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST _____

AGENCY _____

DATE _____

INTOXILYZER SERIAL # _____

- () 1. Ensure that gas tank is attached. Pour a standard alcohol concentration solution into a clean dry simulator. Ensure that a temperature of 34°C ± 0.2°C is reached.
- () 2. Intoxilyzer 8000 display ready.
- () 3. Go to the "Control Testing" menu and press ENTER.
- () 4. Air blank completed.
- () 5. Calibration check complete.
- () 6. Air blank completed.
- () 7. Remove printed record. At _____

INTOXILYZER 8000
Location: YCSO SPARE
Serial Number: 80-000289
Firmware Version: 8105.46
09/01/2014 09:52:49

Standard Lot#: 3391310043
Last Changed By: J. MURRAY5597 #N56

QAS: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Operator: J. MURRAY5597 #N56
YAVAPAI SHERIFFS

Test g/210L Time

Air Blank 0.000 09:53:27
0.100 Cal Check 0.094 09:53:47
Air Blank 0.000 09:54:17

alcohol concentration solution _____ AC.

a clean dry simulator and assemble the simulator and allow temperature to reach 34°C ±

"W" for wet control test. After selection is

SIGNATURE _____

DPS Form Exh G-2 (Rev 05-01)

Preventative Maintenance
Performed -- Timer Reset